Bruksanvisning/Manual

Eurovema

Creates mobility and ergonomics.

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CONTENTS

Contents	2
Introduction	3
Contact	3
Assembly	4
Transportation	4
Safety	5
Over view	6
Brakes	7
Back rest	8
Seat height - gas	9
Seat height - electric	9
Seat angle - manuel	10
Armrest	11
Leg rests	11
Rotation	12
Charging of batteries	13
Technical date	14
Warranty	15
CE-label	15
Maintenance	15

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INTRODUCTION

We congratulate you to the choice of work chair ABC. We hope you will be satisfied with this product from Eurovema AB, constructed and build in Gislaved, Sweden. N-7000 ABC is developed to meet very high demands on comfort and function. The chair is suitable for disabled people and for those who spend a lot of time in their chair. The chair is fully adjustable in order to meet all the needs for adaptation. Read the manual carefully before use.

CONTACT

If you have questions or need help with your product, please contact the local agent. To get in touch with the manufacturer see info below:

Eurovema AB
Tel: +46 371 390 100
Fax: +46 371 189 82
E-post: info@eurovema.se
Baldersvägen 38
SE-332 35 Gislaved
Sweden
www.eurovema.se

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ASSEMBLY

- Open the packing and check that no transport damage has occurred.

- Also check that the delivery harmonize with the order.

The chair is delivered with backrest and armrest not mounted!

- Mount the backrest and the armrests as shown in the picture. Tighten knobs (1 and 2).

TRANSPORTATION

During the transportation of office chair so you can easily remove the back and lower the armrests and lower seat lift to thereby reduce the transport size.

Lower the armrest to the bottom position. Lower the seat lift to the bottom position.

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SAFETY

- Carefully read the manual before using the wheel chair.
- The work chair is designed to be used on even surface in normal indoor climate.
- Be careful when adjusting the high backrest and neckrest seated. Could cause a risk of falling backwards.
- Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.
- Immediately contact service if damages or change in function of the chair is discovered.
- If the chair is used frequently it should have service regularly.
- Make sure to tighten all screws, knobs and levers after adjustments.
- Immediately contact service if damages or change in function of the chair is discovered.
- Service and maintenance should be performed by educated staff.
- Use only original parts from Eurovema at repair.
- Do not exceed maximum user weight.
- Avoid contact with a damage or leaking battery. Used batteries should be recycled.

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OVER VIEW

- 1) seat
- 2) armrest
- 3) back support - standard
- 4) armrest height adjustment
- 5) rear castors - with brake
- 6) depth adjustment - upper back
- 7) armrest width adjustment
- 8) seat height adjustment (right side)
- 9) castors
- 10) seat angle adjustment
- 11) back support height adjustment
- 12) brake controller
- 13) rotation (left side)
- 14) foot plate
- 15) back angle

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BRAKES

Brake

- Push the brake lever (2) forward/down. The rear wheels are locked.

Release

- Pull the brake lever backward/up (2).

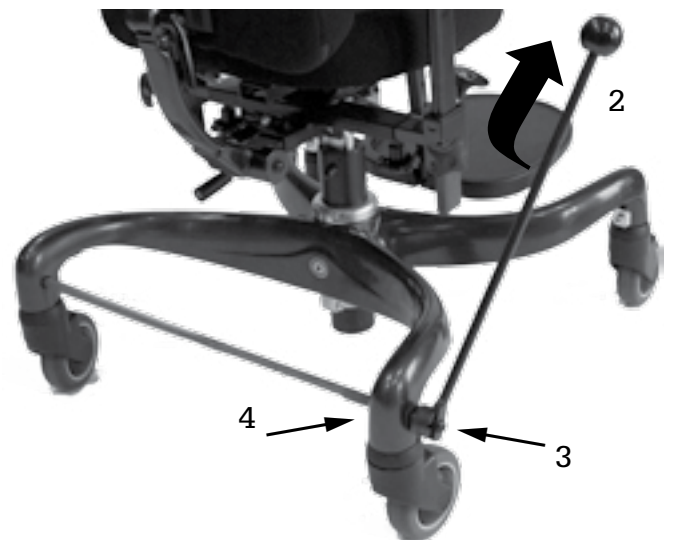
The brake lever can be fitted on the right side (standard) or the left.

Loosen screw (3) and pull out the hexagon bar (4). Enter the bar from the other side. Mount in reverse order.

RISK OF FALLING

Always engage the brakes before entering or leaving the chair. Make sure that the chair is not standing on loose carpets or similar. This can affect the brake ability in a negative way.

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BACK REST - adjustments**Angling of back rest**

- Loosen lever (3). Set the desired angle.
Tighten the lever.

Setting the height of the back rest

- Loosen the knobs (4). Set the desired height.
Tighten the knobs.

Setting the depth of the back rest

- Adjust the depth roughly by removing the screw (1). Choose among four fixed positions. - Finely adjust depth by loosening the screw (2). Tighten the screws (1 and 2) when the desired angle has been reached. Tighten the screw.

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**SEAT HEIGHT - gas****Raising**

- Pull the gas-lift mechanism lever (1) upward. Do not load the seat. Release the lever at the required seat height.

Lowering

- Sit on the chair. Pull lever (1) upward. Release the lever at the required seat height.

Gas lift is set to standard size. To change the default setting, loosen the screws (4). Important to check the stability of the changing dimensions.

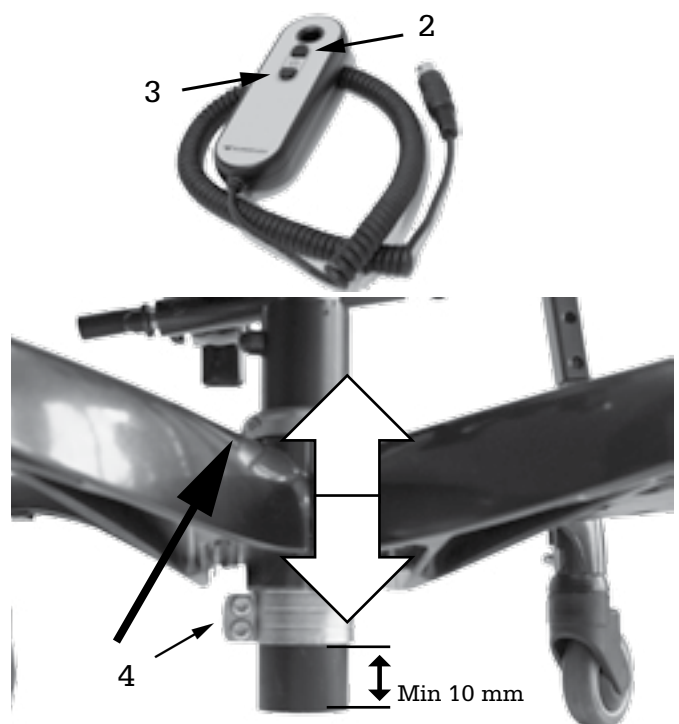
SEAT HEIGHT - electric

If the chair is equipped with an electric seat height the seat height is adjusted by the control box. Push button (2) to increase seat height. Push button (3) to reduce seat height. Release the button when desired height is reached.

Control unit can be placed on the right or left side.

RISK OF GETTING PINCHED

NOTE! When the screws (4) is loosen the seat unit and the gas spring will fall downwards. Put something between the gas spring and the floor to prevent this.

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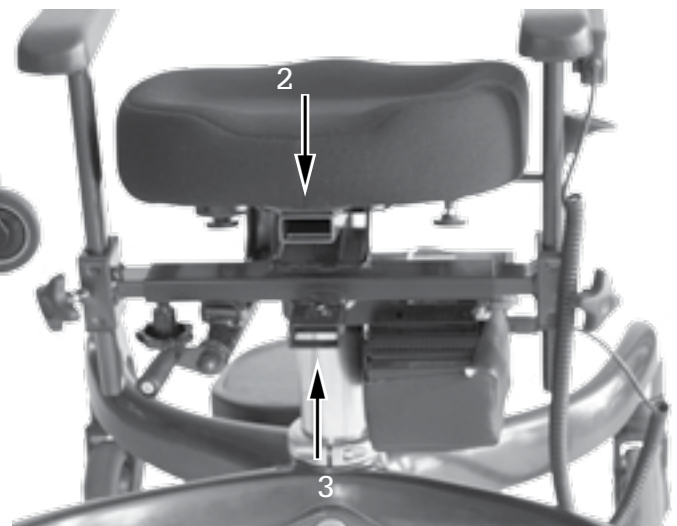
SEAT ANGLE - manual**Forward angling**

- Clockwise CRANKSHAFT AND RELATED PARTS on the control (1). Max. 20° forward.

Backward angling

- Counterclockwise CRANKSHAFT AND RELATED PARTS on the slider. (1). Max 20° backwards.

Seat Cross has two back fixings. Upper attachment (2) used in the seat angle back accompanies seat (tilt). Lower mount (3) only seat angle back are not included.

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**ARMREST - adjustments****Height**

- Loosen the knob (1) and set the desired height. Tighten the knob.

Width

- Loosen the knob (2) and set the desired width. Tighten the knob.

LEG RESTS – single foot plate**Depth**

- Loosen the lever (5). Adjust to the desired depth. Tighten the screw.

Length

- Remove the screw (4) entirely. Set the desired length. Replace the screw and tighten.

Angle (foot plate)

- Turn the screw (3) clockwise to decrease the angle, counter clockwise to increase.

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ROTATION (option)

At the rotary (3) requires the chair to be braked. For the locking lever down (4) to apply the brakes. For the locking lever (4) up to release the brake. Slide the rotating lever (3) downward, the seat can now be rotated 180°. The seat can be locked in three fixed positions. For rotating the lever up to lock into position. The lever (3) can also be mounted on the right side.

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
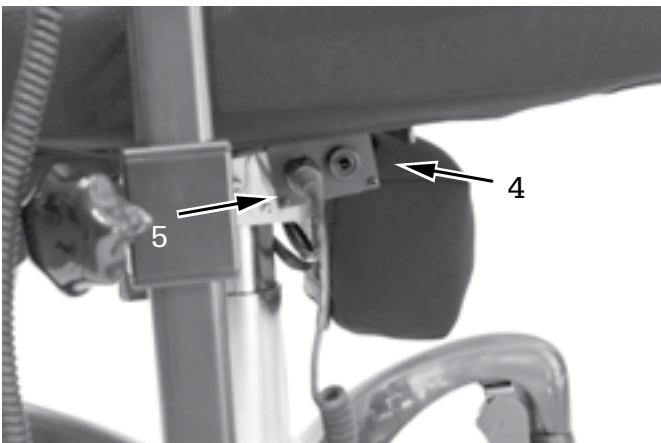
CHARGING OF BATTERIES

To keep the full power of the battery as long as possible it is important to charge it regularly. Charging it once a week is often sufficient, as a suggestion at night when the chair is not in use. Never stop the charging until its completed. Charging can take place indoors in the living area without problems.

When stored, the battery should be charged once a month to prevent it to go under the critical level when the charger cannot start. Do not leave the charging on for long continuous periods as this is not good for the battery. Don't leave the charger plug (2) in the socket (4) when charging is completed. It blocks the electric seat functions. Connector for control box (5).

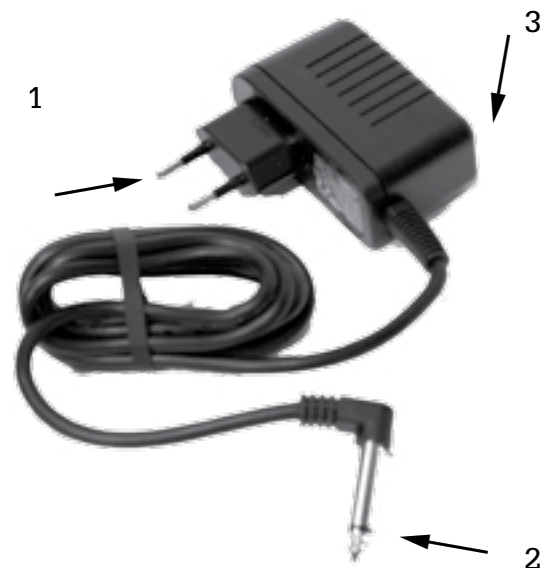1. Connect the charger plug (2) to the socket (4) on the chair. LED (3) lights green.
2. Connect the charger (1) to the wall socket. The LED lights red.
3. When charging is completed the LED (3) lights green.
4. Pull the charger (1) from the wall socket first and then the charger plug (2) from socket (4).

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TECHNICAL DATE

ISO	
chassis, width	
chassis, length	
weight	
user weight (max)	
wheel, diameter	
seat width - between armrests	
seat depth, front seat edge to back	
seat height - std gas spring	
seat height - long gas spring	
seat height - short gas spring	
seat height - actuator short	
seat height - actuator long	
seat sizes, cm	
seat angle - manual	
seat tilt	
back, size - std WxH	
armrest, height	
armrest, plate, size	
upholstery, color	

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*Adjustable between min and max (actuators).

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WARRANTY

We give a 2 year warranty on our products for fabrication fault / damage. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

CE - label 

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. Tested and approved according to Directive NSEN1335-1, -2, -3.

MAINTENANCE

Lift actuator: The gas and electrical lift column (interior tube) can stick and move jerkily. Raise the seat to its highest position and dry off the column properly with a cloth. Then lubricate it with Teflon-based grease.

Fuse: Located next to the battery behind the protective housing see lower picture. The fuse is 15A. If the chair stops suddenly, this can be due to the fuse having blown. Replace the fuse with a new one.

Upholstery: The upholstery is made of 100% polyester and withstands spilling. It withstands high temperatures (60° machine washing – cover). Remove any spots with warm soapy water and a damp cloth.

NOTE! Ensure that all locking wheels and screws are well tightened when adjustments to the chair have been made.

Eurovema
Creates mobility and ergonomics.