Bruksanvisning/Manual

Miniflex Classic, modell 591-33
Flexmobil Classic, modell 691-33

Eurovema

Creates mobility and ergonomics.

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CONTENTS

Contents	2
Introduction	3
Contact	3
Assembly	4
Transportation	4
Safety	5
Over view	6
Electronics	7
Back rest - seat depth, angle	8
Back rest - height	9
Seat angle - manual	10
Seat angle - power	11
Seat height	12
Arm rest - height, width, fold back	13
Controlbox - adjustment	14
Neckrest	15
Leg rest - devided footplate	16
Driving	17
Disengagement	18
Automatic Fuse	19
Charging of batteries	20
Service	21
Warranty	21
CE-label	21
Cleaning	22
Function	22
Storage	22
Technical date	23
Fault trace	24

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INTRODUCTION

We congratulate you to the choice of powerchair. We hope you will be satisfied with this product from Eurovema AB, constructed and build in Gislaved, Sweden. Miniflex/Flexmobil is developed to meet very high demands on comfort and function. The powerchair is suitable for disabled people and for those who spend a lot of time in their chair. The chair is fully adjustable in order to meet all the needs for adaptation. Read the manual carefully before use.

CONTACT

If you have questions or need help with your product, please contact the local agent. To get in touch with the manufacturer see info below:

Eurovema AB
 Tel: +46 371 390 100
 Fax: +46 371 189 82
 E-post: info@eurovema.se
 Baldersvägen 38
 SE-332 35 Gislaved
 Sweden
 www.eurovema.se

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ASSEMBLY

- Open the packing and check that no transport damage has occurred.
- Also check that the delivery harmonize with the order.
- The chair is delivered with backrest and armrest not mounted!
- Mount the backrest and the armrests as shown in the picture. Tighten knobs (1, 2).

TRANSPORTATION

When transporting Miniflex or Flexmobil in a motor vehicle it is important that the brakes are engaged. See section "Disengagement of brakes". The chair should be secured with tie down straps.

Special brackets to be mounted on the chair are available as accessories. The wheelchair is not designed for transport with the user seated in the chair. You can reduce the transport size by removing the armrest, backrest and the legrest. The batteries in the wheelchair are maintenance free and of the type GEL and approved for transport in aircrafts.

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SAFETY

- Carefully read the manual before using the wheel chair.
- The work chair is designed to be used on even surface in normal indoor climate.
- Be careful when adjusting the high backrest and neckrest seated. Could cause a risk of falling backwards.
- Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.
- Immediately contact service if damages or change in function of the chair is discovered.
- If the chair is used frequently it should have service regularly.
- Make sure to tighten all screws, knobs and levers after adjustments.
- Immediately contact service if damages or change in function of the chair is discovered.
- Service and maintenance should be performed by educated staff.
- Use only original parts from Eurovema at repair.
- Do not exceed maximum user weight.
- Avoid contact with a damage or leaking battery. Used batteries should be recycled.

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
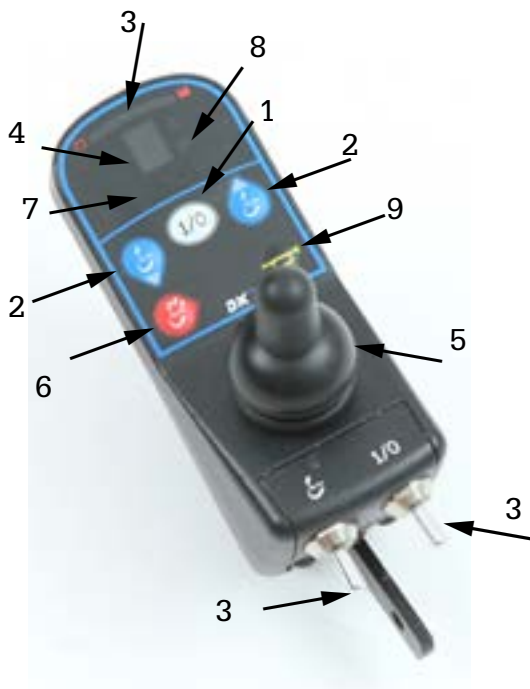
G90T

ELECTRONICS

Miniflex and Flexmobil are power wheelchairs equipped with Dynamic control system to control the power from the batteries out to the motors. The wheelchair and its power assisted seat functions is controlled by the control box, available in three different models; G90T och SHARK. The joystick is for driving the wheelchair in the desired direction. The electronics can be programmed to suit the users needs but usually the original program is fully sufficient. Should a fault occur with the electronics the status LED (8) will blink. Thru counting the number of blink you can deduce the problem. See chap. fault trace.

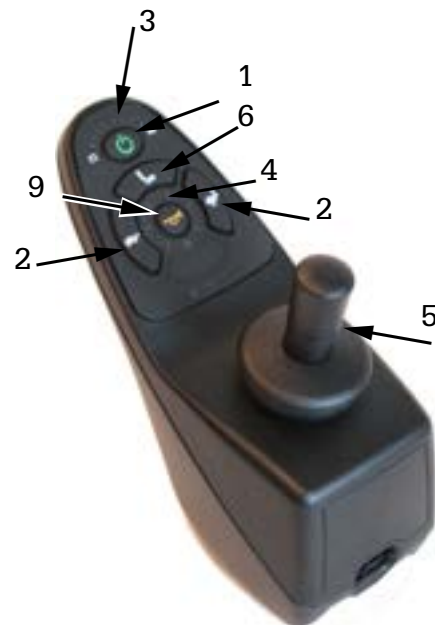- 1) on/off
- 2) choice of drive programs
- 3) battery indicator
- 4) display for drive programs/speed
- 5) joystick
- 6) seat functions
- 7) symbols for seat functions
- 8) status LED
- 9) horn

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BACK REST - seat depth

To adjust the seat depth you can move the back support forward or backwards. Use the wheel nut (1). Adjust the desired seat depth by moving the back support forward or backwards. Lock by moving the lever to its original position.

BACK REST - angle

To adjust the angle on the back support, loosen the lever (2). Adjust to desired angle and lock by moving the lever back to its original position.

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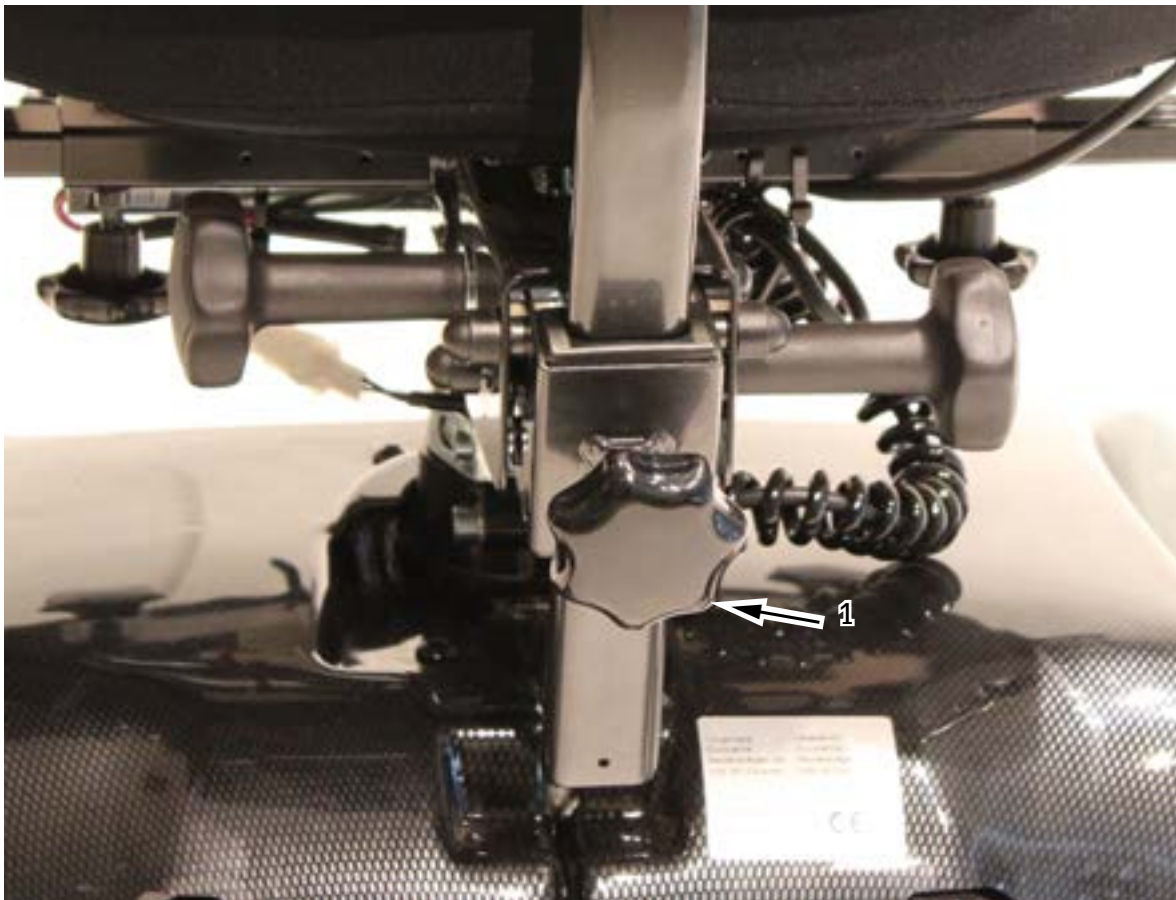
english

BACK REST - height

Loosen the lever (1) by turning it anticlockwise $\frac{1}{2}$ turn. Adjust to desired height of the back support and lock turning the lever clockwise $\frac{1}{2}$ turn.

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SEAT ANGLE - manual

The seat angle is possible to adjust forward 0° to -20°. To adjust the angle loosen the lever (1) pushing in the direction of the arrow. Adjust to desired angle and lock by moving the lever to its original position.

**RISK OF FALLING!**

NOTE! When the lever (1) is pushed down the seat angle mechanism is loose and the seat can tilt forward rapidly if you are seated. Be careful when adjusting the seat angle.

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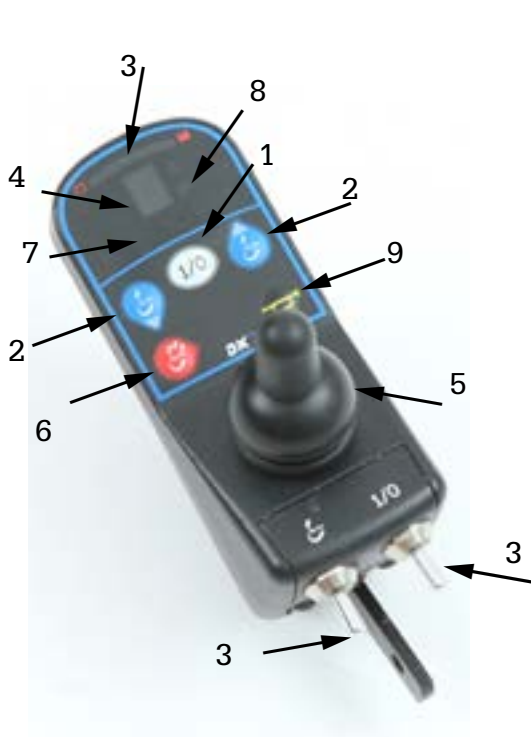
G90T

SEAT ANGLE - power

If the chair is equipped with power assisted seat tilt it's activated by the controlbox. Push button "6".

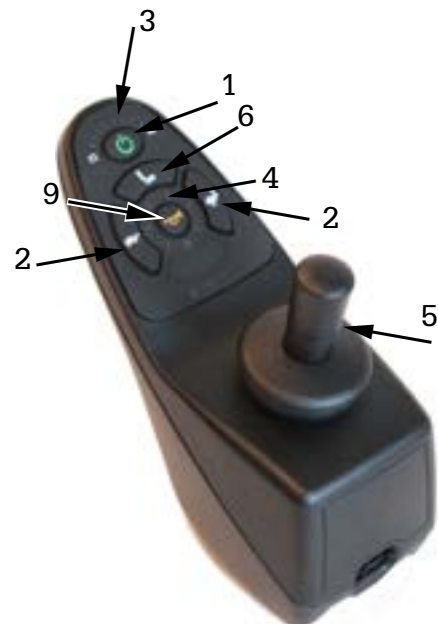
G90T

Choose a seat symbol (7) for the seat angle by moving the joystick (5) to one side to the second icon from the left side blinks.

SHARK

Choose a seat symbol (6). Press the seat symbol (6) and select the number "2".

Move the joystick forward to activate the seat tilt forward and backwards for backwards. The seat tilt stops automatically when the endpoints is reached. Push button (2) to get back into drive mode and the choosed drive program will be visible in the display (4).

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SEAT HEIGHT

Start the electronics of the wheel chair by pushing button (1) on the control box. Then push button (6).

G90T

Choose a seat symbol "7" and pull the joystick to the side to select the fourth icon from the left. For seat lift thro leaning the joystick (5) side ways until the symbol blink.

Shark

The nubmer "1" lights up on button (6). Move the joystick forward to activate the seatlift up and backwards down. The seat lift stops automtically when the endpoints is reached. Push button (2) to get back into drive mode and the choosed drive program will be visble in the display (4).

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ARM REST - height and width

To adjust the width between the arm supports, loosen the wheel nut (1). Adjust to desired width and tighten the wheel. Repeat on the other arm support. Tighten the wheel nuts. To adjust the height of the arm support, loosen the wheel lever (2). Adjust to desired height and tighten the wheel lever.

**OPTION!****ARM REST - fold back**

It is possible to fold the arm support backwards to make sideways transfer easier or to get closer to a table. Push lever (3) in the direction of the arrow to fold the arm support backwards. To get it back to upright position, use your hand to lift it up where it locks automatically.

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CONTROLBOX - Adjustments

The controlbox position can be adjusted by loosening the screws 1,2 and 3 with allen key nr 5. Loose screw 1 to adjust in depth and screw 2 and 3 to adjust angle and height. Adjust to desired position and tighten the screws. The controlbox can also be swung away when you would like to get close to a table i.e.

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**OPTION!****NECK REST - adjust**

If the chair is equipped with a neck support its adjusted by the screw and wheels (1, 2, 3) to adjust to desired position. Move the levers back to the original position to lock.

**RISK OF FALLING!**

NOTE! When the lever (1 and 3) is opened the neck support come loose and can rapidly fall backwards if you are seated in the chair. Be careful when adjusting the neck support!

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**LEG REST - single footplate**

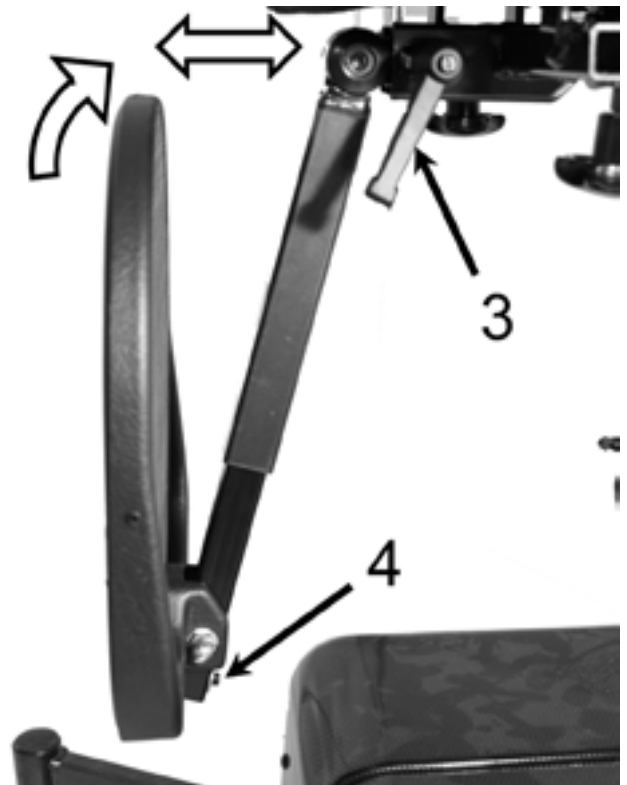
If the chair is equipped with a single footplate you can adjust the height and lower leg length by loosening screw (1 and 2) with allen key 5 mm. Adjust to desired position and tighten the screws. To adjust depth and the angle of the footplate loosen screws (3 and 4). Adjust to desired position and tighten the screws. The footplate is possible to fold up to make it easier to get in and out of the chair.

OPTION! (no picture)

LEG REST - divided footplate

If the chair is equipped with divided footplates it's adjusted in height and lower leg angle by loosening screws. Adjust to desired position and tighten the screws. To adjust depth and angle of the footplate loosen screw. Adjust to desired position and tighten the screws. To adjust the legrest position sideways loosen the knob. Adjust to desired position and tighten the knob. The footplate is possible to fold up to make it easier to get in and out of the chair.

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DRIVING

Place yourself as comfortable as possible in the chair and let the arm you should use to control the wheelchair rest on the armrest so that your hand can grip the joystick (5) comfortable. Start the electronics by pushing button (1), wait a couple of seconds until the batteryindicator (3) has stopped twinkle.

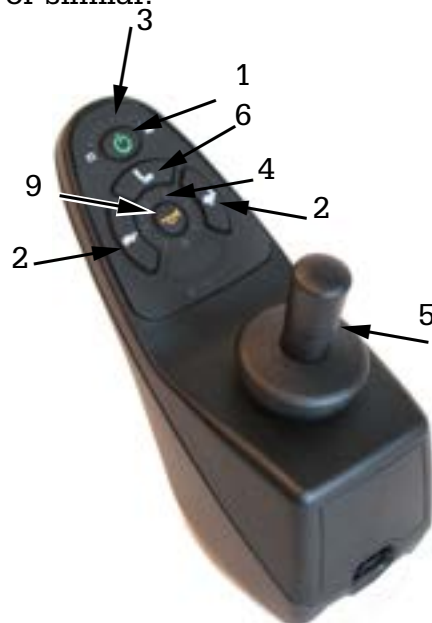
G90T/SHARK

Choose desired driveprogram (1,2 or 3) with the buttons (2), where 3 is the fastest. The more LEDS the faster the wheelchair goes. Now lean the joystick at the direction you wish to go and the wheelchair goes there. Release the joystick and the wheelchair will stopp immediately. This wheelchair is intended for indoor use only and designed to climb obstacles not higher than 3 cm. These obstacles should be passed with the front wheels parallell to the obstacle.

RISK OF INJURE!

NOTE! Make sure that you have a lot of free space around the wheelchair when driving the first time. Practice to drive backwards, turning and forward at different speed to get familiar how the wheelchair behave at different situations. Be careful when passing obstacles like thresholds or similar.

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DISENGAGEMENT OF BRAKES

To be able to transport the wheelchair without using the electronics you need to disengage the brakes/motors. This is done by turning off the electronics and lift the red lever placed on the chassi leg by the rear castor (1). Now you can transport the wheelchair without motor brakes. To get back into driving you fold the red lever back to its original position and starts the electronics.

If you start the electronics when the brakes are disengaged it will cause an error code on the controlbox. Turn off the electronics engage the brakes and restart the electronics, the error code will disappear.

RISK OF INJURE!

NOTE! Make sure that the wheelchair is on flat ground when disengaging the brakes. Otherwise the wheelchair can start rolling away without control.

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AUTOMATIC FUSE

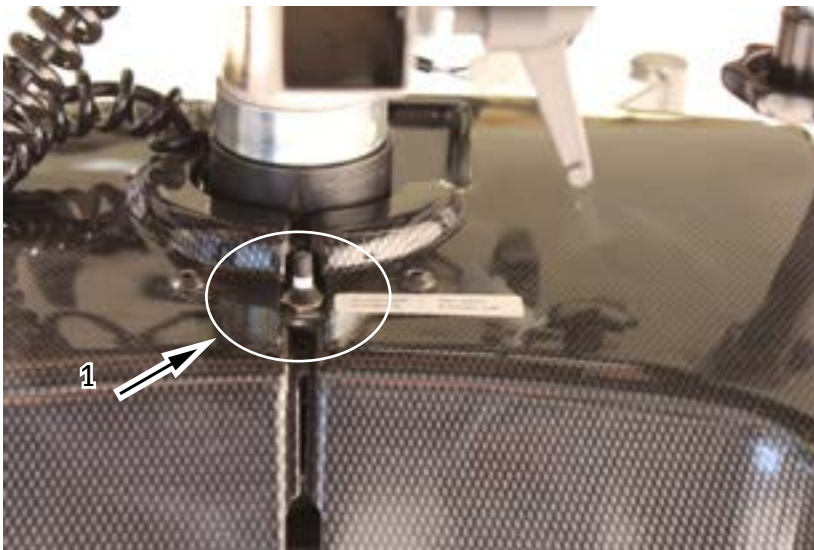
The automatic fuse is placed on the rear cover on Flexmobil (2) and in the front between the covers on Miniflex (1). The fuse will disengage if a severe electrical failure occurs on the chair. The button pops out approx. 8 mm and a white line becomes visible.

The automatic fuse can be used as main power switch. When the wheelchair is stored for a longer period the fuse should be disengaged.

NOTE!

When the batteries is charged the fuse must be engaged. For transport in a aeroplane the batteries usually must be disconnected and for this you can use the automatic fuse that should be disengaged.

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CHARGING OF BATTERIES

To keep the full power of the batteries as long as possible it is important to charge them regularly, as a suggestion at night if the chair is used during daytime. Never stop the charging until its completed. Charging can take place indoors in the living area without problems. When stored the batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start. Do not leave the recharging on for long continuous periods as this is not good for the batteries.

NOTE!

If the batteries are damaged or leak, all contact with these should be avoided to prevent corrosive injuries. Contact an authorised workshop as soon as possible! Used batteries should be handed in for recycling.

1. Connect the charger plug to the control box on the chair. Located in the front of the controlbox. See picture.
2. Connect the charger to the wall socket.
3. Check on the charger's indicator lamps that recharging has started. Information on the charger is supplied with the charger.
4. When recharging has been completed, switch off the charger or remove the plug from the wall socket and then the charger plug from the control box.

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SERVICE

If the wheel chair is used frequently, i.e. every day, it ought to have service once a year. This to control that the function and security of the wheel chair is kept through all its lifetime. If service is needed contact primarily your local dealer.

WARRANTY

We give a 2 year warranty on our products for fabrication fault/damage. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

CE - label

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. Tested and approved according to directive SSEN1335–1,2, 3. Firetested according to ISO7176-16.

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CLEANING:

Wipe the surface of the wheelchair with a damp cloth with a mild detergent. Place the chair on plain surface for drying. The upholstery can be cleaned with any existing brands on the market. Covers can be machine washed in 60 degrees Celsius.

FUNCTION:

If loose screws or loose parts is discovered somewhere on the wheelchair or a change in drive characteristic, immediate repair should take place when this could affect safety. Contact your local agent or the manufacturer for help and questions.

Be sure to tighten all screws and knobs after seat adjustments.

STORAGE:

If the wheelchair should be unused for a longer period (a month or more) it should be stored in a clean, dampfree room tempered space. The batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start.

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TECHNICAL DATA

	Fakta. Data	
Klass	A	Class
ISO	122127	ISO
Chassi	Miniflex/Flexmobil	Chassis
Drivning Miniflex	Framhjul. Front wheel	Drive Miniflex
Drivning Flexmobil	Centralhjul. Central wheel	Drive Flexmobil
Hindertagning	3 cm	Obstacle climbing
Markfrigång	6 cm	Ground clearance
Totallängd - inkl. fotplatta	Miniflex 90 cm Flexmobil 106 cm	Total length - incl. footplate
Totallängd - chassi	Miniflex 72 cm Flexmobil 82 cm	Total length - chassis
Totalbredd - chassi	Miniflex 59 cm Flexmobil 58 cm	Total width - chassis
Totalvikt	Miniflex 54 kg Flexmobil 74 kg	Total weight
Vändutrymme	Miniflex 120 cm Flexmobil 96 cm	Turning space
Drivhjul	Miniflex 20 cm Flexmobil 22,5 cm	Drive wheels
Länkhjul	12,5 cm	Castors
Hastighet-max	4 km/h	Speed - max.
Körtid/laddning - max.	8,5 h	Driving time / charge - max.
Batterier	Miniflex 2x12V-20Ah Flexmobil 2x12V-25Ah	Batteries
Elektronik	Dynamic	Electronics
Sits BxD	40x40, 40x43, 40x46, 46x46, 49x51 cm	Seat, sizes WxD
Sitsbredd mellan armstöd	40-59 cm	Seat, width between armrest
Sitsdjup, från rygg till sits framkant	29-54 cm	Seat, depth -from back to seat front
Sitshöjd - kort ställdon, Miniflex	43-61 cm +5 cm *	Seat, height - short actuator
Sitshöjd - kort ställdon, Flexmobil	42-60 cm +5 cm *	Seat, height - short actuator
Sitshöjd - långt ställdon, Miniflex	49-74 cm +5 cm *	Seat, height - long actuator
Sitshöjd - långt ställdon, Flexmobil	48-73 cm +5 cm *	Seat, height - long actuator
Sitsvinkling, manuell, framåt	0° - -20°	Seat, angle, manual, forward
Sitstilt, el, framåt-bakåt	-14° - +17°	Seat, tilt, power, forward-backward
Ryggstöd, storlek, BxH	35x30, 35x44, 40x30, 40x44 cm	Back, size, std WxH
Ryggstöd, höjd från sits	43-62 cm	Back, height from seat
Nackstöd, storlek	21x23 cm	Neckrest, size
Klädsel, färger	Svart. Black	Upholstery, color
Armstöd, höjd	19-25 cm	Armrest, height
Armstöd, platta, storlek	31x8 cm	Armrest, plate, size

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*Adjustable between min and max (actuators).

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english

FAULT TRACE

Problem?	Action!
The wheelchair doesn't start?	
The automatic fuse is released?	Check the fuse - see the manual.
The batteries is discharged?	Charge the batteries!
Loose cables?	Contact service!
The batteries can not be charged?	
The automatic fuse is released?	Check the fuse - see the manual
The batteries is totally discharged?	Contact service!
The charger is broken?	Contact service!
Battery indicator drops quickly despite charged batteries?	
The batterier starts getting bad?	Contact service!
Error in the electronics?	Contact service!
The status LED is blinking on the controlbox?	
Electronics has been started when brakes are disengaged?	Turn off and engage the brakes- see the manual!
An electric fault has occurred on the wheelchair?	Contact service!
The wheelchair is running slow?	
Is the right drivprogram selcted?	Select a quicker driveprogram- see the manual!
The wheelchair don't manage to climb obsticles?	
A too slow driveprogram is selected?	Select a quicker driveprogram- see the manual!

Eurovema
Creates mobility and ergonomics.