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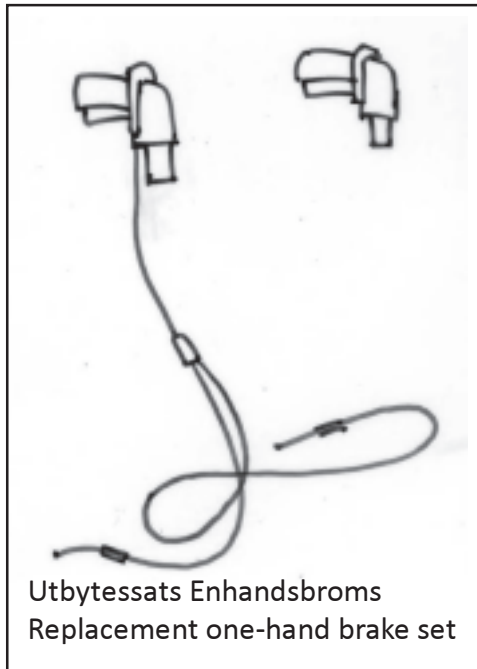
One-hand brake for Volaris S7,
Volaris S7 Smart
and Volaris S7 Patrol

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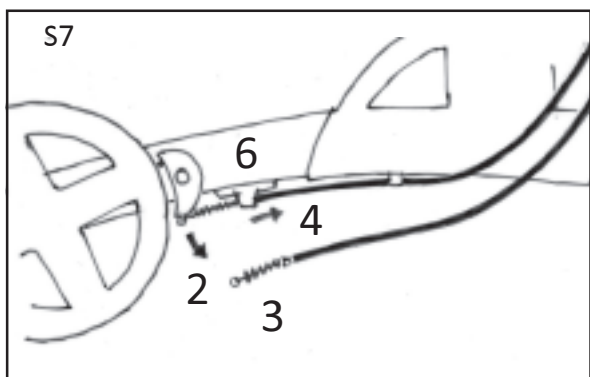
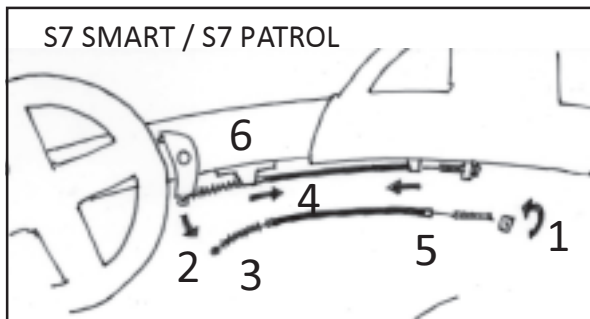
The purpose of the one-hand brake is to enable users who have difficulty in braking with both hands to brake the rollator safely. The one-hand brake handle is fitted on the side where the user has sufficient strength and movement in the hand to perform a braking action. After assembly, activation of either the travel brake or the parking brake should lock both rear wheels.

IT IS VERY IMPORTANT THAT THE BRAKE IS ADJUSTED AT A HANDLE HEIGHT APPROPRIATE FOR THE USER AND THAT THIS HANDLE HEIGHT REMAINS THE SAME WHEN THE ROLLATOR IS IN USE.

If the handle is subsequently lowered or raised there is a risk that the brake will not function. Assembly of the one-hand brake varies between different rollators models. Therefore read the instructions very carefully. The entire assembly should be carried out with the brakes unlocked.

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Detach the wire from the brake yoke and wire bracket.

Undo the brake bolt using an Allen key and remove the old brake yoke. Replace it with the new brake yoke for the one-hand brake. Perform the same procedure on both sides.

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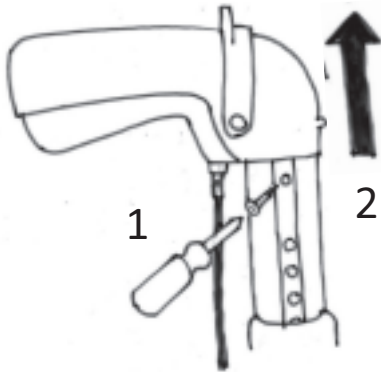
Dismantling

1. Detach the adjustment screw from the nut.
- 2-3. Detach the wire from the brake yoke by first unfastening the spring from the brake yoke.
4. Pull the wire covering forwards until it passes through the bracket.
5. Remove the whole wire by pulling it backwards.
6. NOTE! The wire bracket can be moved about 5 mm alongside the profile. To assemble the one-handbrake the wire bracket must be positioned next to the rear wheel.
7. Perform the same procedure on the opposite side.

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
First perform the procedure on the side of the rollator where the brake will be operated

1. Undo the screw using a screwdriver
2. Unfasten the handle by pulling it upwards.

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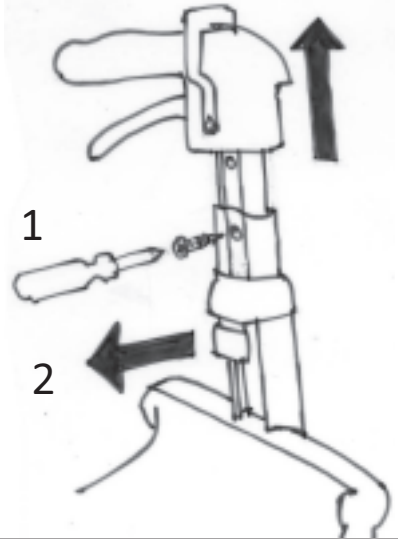
First perform the procedure on the side of the rollator where the brake will be operated.

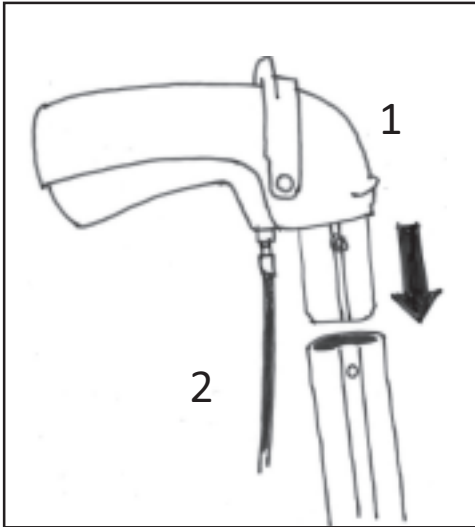1. Undo the screw using a screwdriver.
2. Pull out the handle adjustment button at the same time as you pull the whole handle upwards until it is unfastened. NOTE! Secure the upper handle profile with tape or suchlike so that it does not slide down into the lower profile.

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
Assembly

1. Place the one-hand brake handle with wires on the side where the brake will be operated. The handle should point backwards.

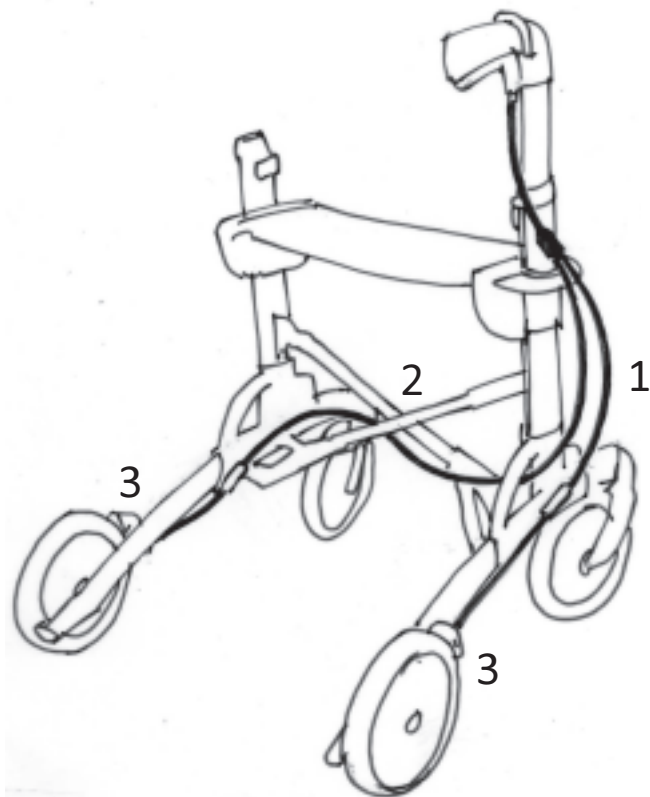2. Attach the handle using the screw and screwdriver.

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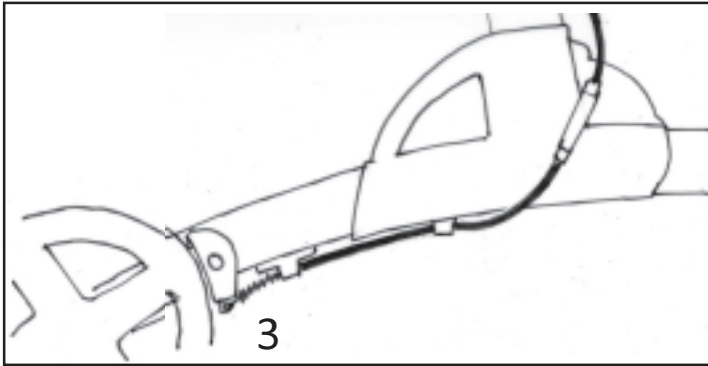
1. The short section of wire should run along the outside of the rollator. First run it through the wire bracket and spring and then attach it to the brake yoke.

2. Run the long wire under the grey plastic arch and then around the cross profile as shown in the illustration.

3 Run the wire through the wire bracket and spring and then attach it to the brake yoke.

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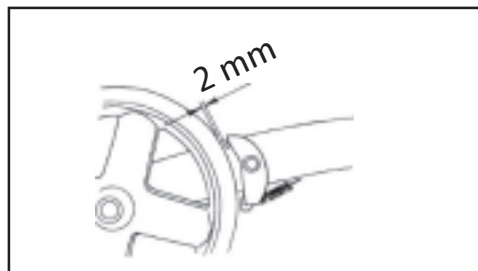
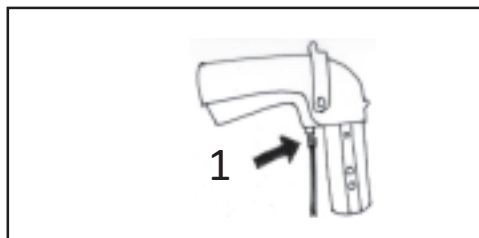
On the side where the brake will not be operated, the handle is replaced with the accompanying handle without wire. Assembly should be carried out according to the previous instruction.

If the one-hand brake is mounted on a Volaris S7 SMART or a Volaris S7 PATROL the original handle can also remain in position if the user finds it more comfortable.

Neither the parking brake nor the travel brake will function regardless of which handle is selected.

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THE BRAKING ACTION FOR THE ONE-HAND BRAKE SHOULD BE ADJUSTED ONCE THE HANDLE HEIGHT IS APPROPRIATELY POSITIONED FOR THE USER.

If the handle height is subsequently adjusted there is a risk that the braking action will also be changed! The adjustment should always be performed with the parking brake disengaged. The brake can be adjusted at three places:

1. at the handle and 2. at the respective section of wire after the dividing point. Adjust the brake so that the brake heel lies approx. 2 mm from the wheel.

Carefully check that the travel brake as well as the parking brake lock the wheels on both sides. Secure loose hanging wires with the accompanying wire ties. How many are used and where they are placed depends on the rollator. Be careful to ensure that the wires are not too tight.

The rollator should be able to unfold and fold.