

Eurovema

Creates mobility and ergonomics.

CE




*Arbetsstol / Workchair
laddningsinstruktion /
Charging of batteries*

LADDNING AV BATTERIER

För att erhålla batteriets fulla prestanda så länge som möjligt är det viktigt att det laddas regelbundet. Oftast räcker det att ladda batteriet en gång per vecka, helst under dagtid när stolen inte används. Avbryt ej laddningen innan den är färdig. Laddning kan ske inomhus i bostaden utan problem. Vid förvaring bör batteriet laddas cirka en gång i månaden för att ej tömmas till den kritiska nivån då laddaren ej längre startar laddningen. Lämna ej laddningen på långa perioder utan avbrott då detta ej är bra för batteriet. Låt ej heller laddkontakt (2) sitta kvar i uttag (4) efter avslutad laddning. Då fungerar ej de elektriska sidsfunktionerna. Uttag för manöverdosa (5).

1. Anslut laddkontakten (2) till uttaget (4) på stolen. Dioden (3) på framsidan av väggkontakt lyser grönt.
2. Anslut laddarens väggkontakt (1) till 220V vägguttag. Dioden (3) på framsidan av väggkontakt börjar lysa rött.
3. När laddningen är klar lyser dioden på framsidan av väggkontakt (3) grönt.
4. Tag ur väggkontakten (1) först och sedan laddkontakt (2) ur uttag (4).

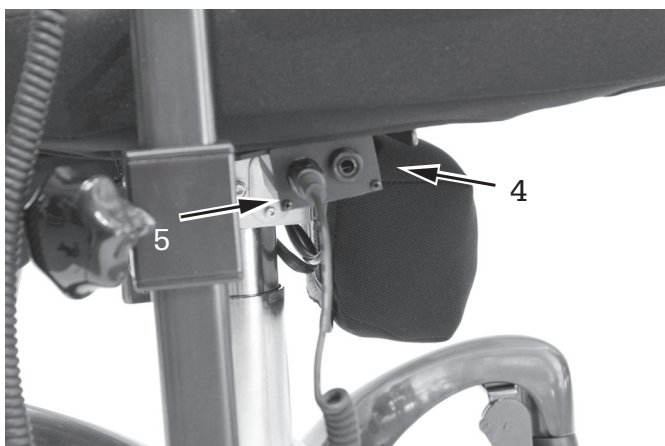

CHARGING OF BATTERIES

To keep the full power of the battery as long as possible it is important to charge it regularly. Charging it once a week is often sufficient, preferably during the daytime when the chair is not in use. Never stop the charging until its completed. Charging can take place indoors in the living area without problems. When stored, the battery should be charged once a month to prevent it to go under the critical level when the charger cannot start. Do not leave the charging on for long continuous periods as this is not good for the battery. Don't leave the charger plug (2) in the socket (4) when charging is completed. It blocks the electric seat functions. Connector for control box (5).

1. Connect the charger plug (2) to the socket (4) on the chair. LED (3) lights green.
2. Connect the charger (1) to the wall socket. The LED lights red.
3. When charging is completed the LED (3) lights green.
4. Pull the charger (1) from the wall socket first and then the charger plug (2) from socket (4).





Eurovema

Creates mobility and ergonomics.

Eurovema Mobility AB

Baldersvägen 38, SE-332 35 Gislaved, Sverige

Tel. +46 371 390 100 | Fax. +46 371 189 82

info@eurovema.se | www.eurovema.se