# bruksanvisning / manual

---

 **EUROFLEX**  
**FORMA**

**Eurovema**  
Creates mobility and ergonomics.

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## INTRODUCTION

We congratulate you to the choice of work chair Forma. We hope you will be satisfied with this product from Eurovema AB, constructed and build in Huddinge, Sweden. Forma is developed to meet very high demands on comfort and function. The chair is suitable for disabled people and for those who spend a lot of time in their chair. The chair is fully adjustable in order to meet all the needs for adaptation. Read the manual carefully before use.

english

## CONTACT

If you have questions or need help with your product, please contact the local agent.

To get in touch with the manufacturer see info below:

Eurovema AB  
Tel: +46 371 390 100  
Fax: +46 371 189 82  
E-post: [info@eurovema.se](mailto:info@eurovema.se)  
Baldersvägen 38  
SE-332 35 Gislaved  
Sweden  
[www.eurovema.se](http://www.eurovema.se)

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## ASSEMBLY

-Open the packing and check that no transport damage has occurred.

-Also check that the delivery harmonize with the order.

The chair is delivered with backrest (1) and armrest (2) not mounted!

Backrest and armrest are equipped with stop screws (3 and 4) to prevent that they come loose when adjusting to the end points. Demands acc. to EN1335 standard. Screws are delivered separately in a small bag.

-Mount the backrest and the armrests as shown in the picture. Tighten knobs (6 and 5)

- Mount the stop screws (3 and 4). Use the outer holes on place 4.

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## SAFETY

- Carefully read the manual before using the wheel chair.
- The work chair is designed to be used on even surface in normal indoor climate.
- Be careful when adjusting the high backrest and neck-rest seated. Could cause a risk of falling backwards.
- Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.
- Immediately contact service if damages or change in function of the chair is discovered.
- If the chair is used frequently it should have service regularly.
- Make sure to tighten all screws, knobs and levers after adjustments.
- Immediately contact service if damages or change in function of the chair is discovered.
- Service and maintenance should be performed by educated staff.
- Use only original parts from Euroflex at repair.
- Do not exceed maximum user weight.
- Check that stop screws are mounted by the armrest and back support hold. They prevent armrest and back support to come loose when adjusting to the end points. Demands acc. to SS-EN 1335 standard.
- Avoid contact with a damage or leaking battery. Used batteries should be recycled.

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## BRAKES

The chair brakes on 2 or 4 castors depending on model. As standard the chair brakes on the rear castors, pic. (3), but could be equipped with front- or 4 wheel brakes as option. Move the lever forward to released the brakes and the chair is possible to roll. The brakes will remain unengaged until the lever is changed to locked position. The lever affects wires that goes through the chassis down to each braked wheel. The brake lever can be mounted on the left or the right side. Pic (1): brake position, pic (2): unbrake position. Pull the black knob to the end point to extend the lever and make the brake motion easier.

## RISK OF FALLING

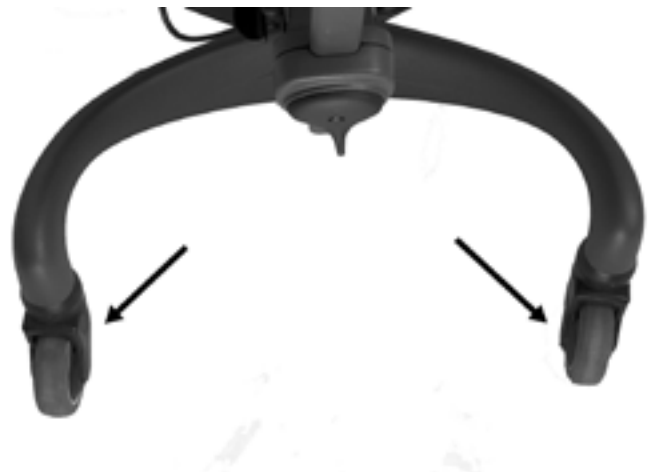
Always engage the brakes before entering or leaving the chair. Make sure that the chair is not standing on loose carpets or similar. This can affect the brake ability in a negative way.

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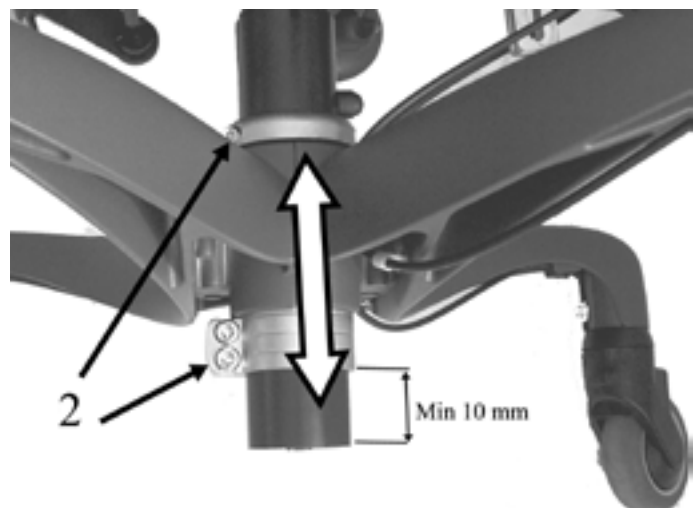
## SEAT HEIGHT - gas

To adjust the seat height use the lever (1). It can be mounted on the left or the right side. Increase the seat height by pushing the lever downwards. The seat should be unloaded. Release the lever when desired height is reached. Reduce the seat height by push the lever downwards. The seat should be loaded. Release the lever when desired height is reached. The gas spring is as standard prepare for the seat height span 45 - 59 cm. (14 cm stroke). By loosen the screws (2), lower pic, the position of the gas spring can be changed. Adjust to desired position. NOTE ! Not less than 10 mm must be visible under the fixation. Tighten the screws firmly. !!

### RISK OF GETTING PINCHED

NOTE ! When the screws (2) is loosen the seat unit and the gas spring will fall downwards . Put something between the gas spring and the floor to prevent this.

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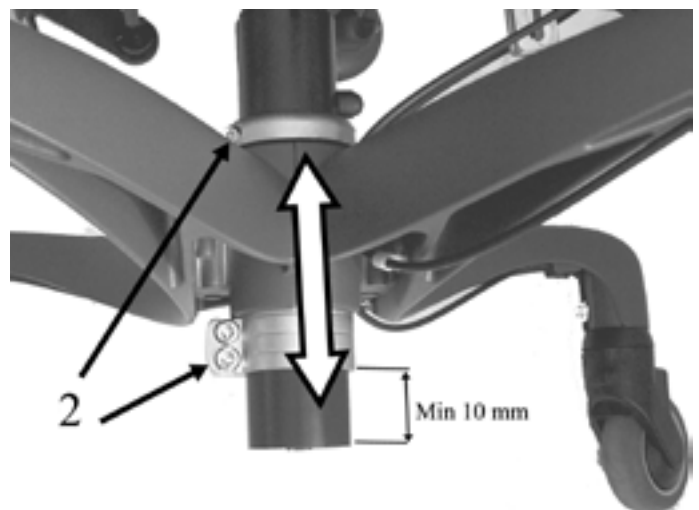
## SEAT HEIGHT - electric

If the chair is equipped with an electric seat height the seat height is adjusted by the control box. Push button (1) to increase seat height. Push button (2) to reduce seat height. Release the button when desired height is reached. With electric seat height its also possible to change position of the actuator in the chassis. See "SEAT HIGHT – gas" above.

## RISK OF GETTING PINCHED

NOTE ! When the screws (2) is loosen the seat unit and the gas spring will fall downwards . Put something between the gas spring and the floor to prevent this.

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## BACK REST - seat depth

To adjust the seat depth you can move the back support forward or backwards. Use the lever (1). Loosen the back support by pulling the lever in the direction of the arrow. Adjust the desired seat depth by moving the back support forward or backwards. Lock by moving the lever to its original position. The lever can also be turned, as the arrow shows, to a position that feels comfortable for your hand. Adjust the back support angle.

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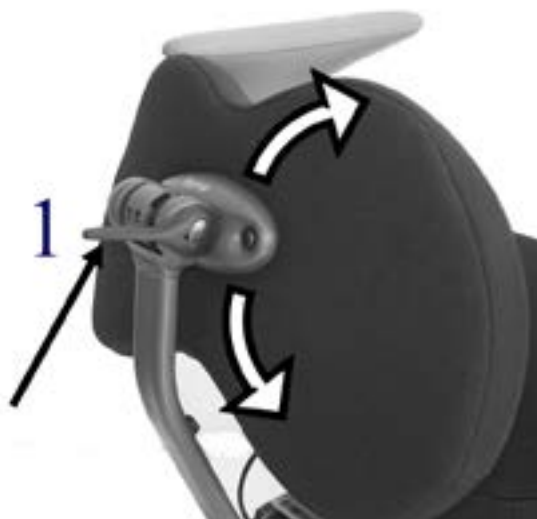
## BACK REST - (std) angle

To adjust the angle on the back support, loosen the lever (1) by pulling it in the direction of the arrow. Adjust to desired angle and lock by moving the lever back to its original position.

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
## BACK REST - height

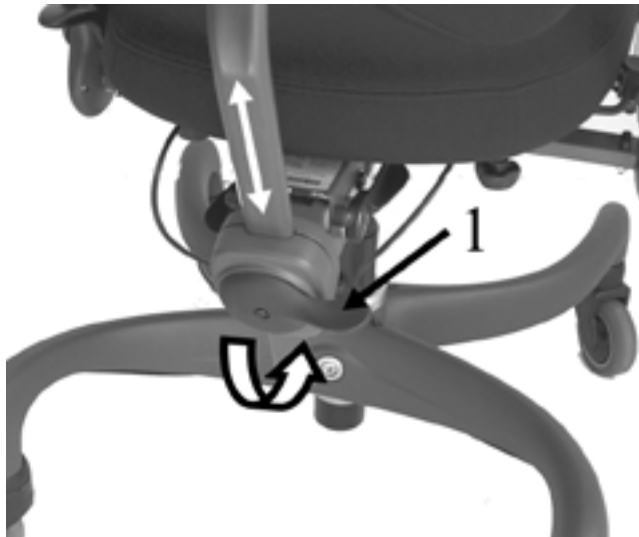Loosen the lever (1) by turning it anticlockwise  $\frac{1}{2}$  turn. Adjust to desired height of the back support and lock turning the lever clockwise  $\frac{1}{2}$  turn.

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## BACK REST - (active, comfort) adjust

If the chair is equipped with an upper back support, model active or comfort, its adjusted by loosening the levers (1 and 2). The left lever (1) adjust the depth and the right lever (2) adjust the angle of the upper support. Loosen the levers by moving them in the direction of the arrows. Move the levers back into the original position when the adjustments are done, to lock..

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**NECK REST - adjust**

If the chair is equipped with a neck support its adjusted by the levers (1, 2, 3). Open the levers and adjust to desired position. Move the levers back to the original position to lock.

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**RISK OF FALLING**

NOTE ! When the lever (1 and 2) is opened the neck support come loose and can rapidly fall backwards if you are seated in the chair. Be careful when adjusting the neck support!

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## SEAT ANGLE

The seat angle is possible to adjust forward 0° - 20°. To adjust the angle loosen the lever (1) pulling it in the direction of the arrow. Adjust to desired angle and lock by moving the lever to its original position. The lever can also be turned, as the arrow shows, to a position that feels comfortable for the hand.

### RISK OF FALLING

NOTE ! When the lever (1) is loosened, the seat angle mechanism is loose and the seat can tilt forward rapidly if you are seated. Be careful when adjusting the seat angle.

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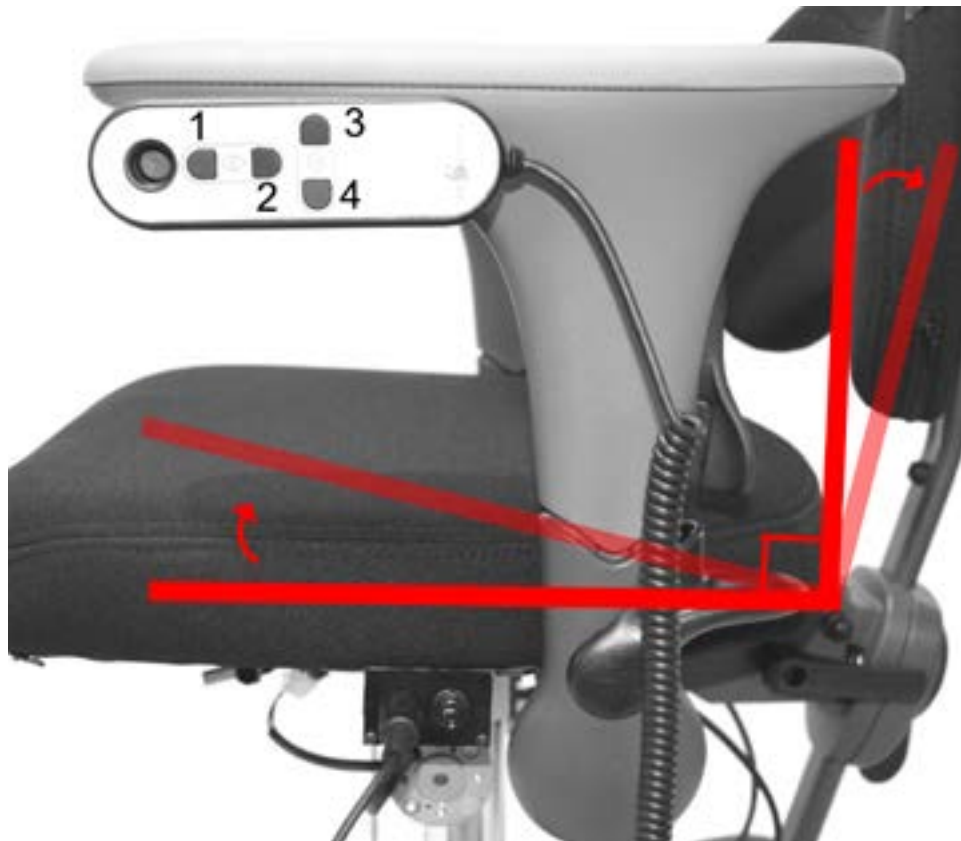
**SEAT TILT - electric**

If the chair is equipped with an electric tilt function, the seat can be tilted forward 20° and backwards 10°. To tilt the seat forward push button (3) and backwards button (4). Release button when desired angle is reached.

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**ARM REST - height and width**

To adjust the width between the arm supports, loosen the wheel nut (2). Adjust to desired width and tighten the wheel. Repeat on the other arm support. Tighten the wheel nuts. To adjust the height of the arm support, loosen the lever (1) in the direction of the arrow. Adjust to desired height. Lock the arm support by pushing the lever back to original position

english

**ARM REST - fold back**

It is possible to fold the arm support back wards to make sideway transfer easier or to get closer to a table. Push lever (3) in the direction of the arrow to fold the arm support back wards. To get it back to upright position, use your hand to lift it up where its locked automatically.

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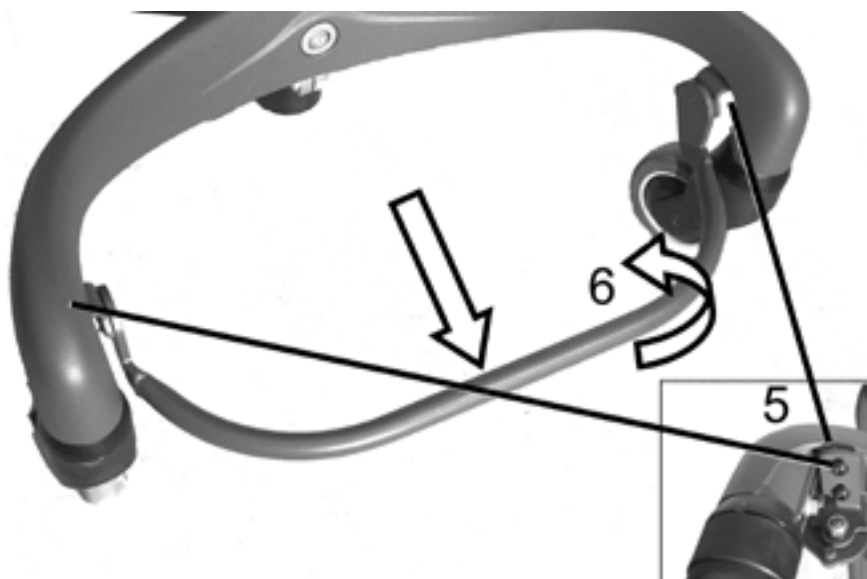
## LEG REST

### Foot ring:

The foot ring is mounted in the prepared holes in front of the chassis. Use four screws (5). The foot ring is also possible to fold up to make transfer easier (6).

### Single foot plate:

If the chair is equipped with a single footplate you can adjust the height and lower leg length by loosening screw (1 and 2) with allen key 5 mm. Adjust to desired position and tighten the screws. To adjust depth and the angle of the footplate loosen screws (3 and 4). Adjust to desired position and tighten the screws. The footplate is possible to fold up to make it easier to get in and out of the chair.

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## MAINTENANCE

To make the chair work properly and be safe during its lifetime it's important to follow these maintenance instructions.

### Cleaning:

Wipe the surface of the work chair with a damp cloth with a mild detergent. The upholstery can be cleaned with any existing brands on the market. Covers can be machine washed in 40 degrees Celsius.

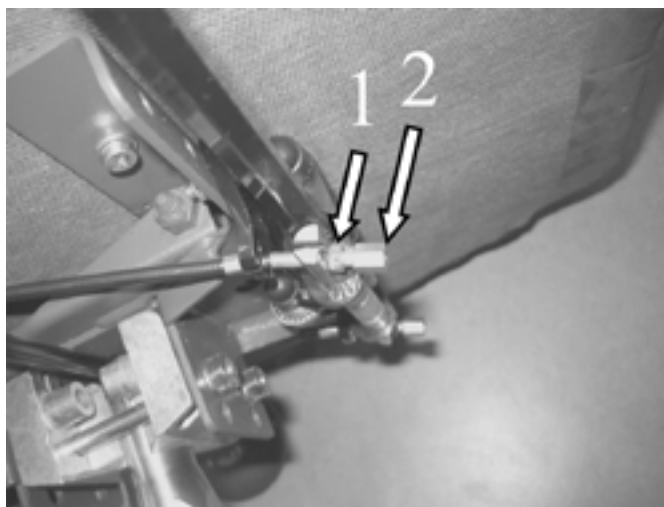
### Gasspring:

The gasspring for the seatlift should be cleaned / lubricated once a year. Raise the seat lift to its highest position and dry clean the rod with cloth. Lubricate the rod with a thin layer of grease.

### Brakes:

If the brakes doesn't work properly it might need some adjustment of the wires. Loosen screw (1) adjust the tension with screw (2) until the brakes works properly. Fasten the screw (1). Repeat on all the wires. It could be 2 or 4 wires depending on model.

If the brakes still doesn't work, contact your local supplier.

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## MAINTENANCE

### Batteries:

To keep the full power of the battery as long as possible it is important to charge them regularly. Charging once a week is often sufficient for normal usage. Never stop the charging until its completed. Charging can take place indoors in the living area without problems.

### Function:

If loose screws or loose parts is discovered somewhere on the chair or a change in characteristic, immediate repair should take place when this could affect safety. Contact your local agent or the manufacturer for help and questions.

Be sure to tighten all screws and knobs after seat adjustments.

### Storage:

If the chair should be unused for a longer period ( a month or more) it should be stored in a clean, dampfree room tempered space. The batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start.

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Minsta transport mått:

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## TRANSPORT

When transporting Forma you can disassemble backrest, fold the armrest and lower the seat lift.

- Loosen the stop screw (see section "assembly") of the backrest and remove the backrest.
- Fold the armrests backwards or remove them. (loosen the stop screws) (see section "assembly")
- Load the seatlift and lower it to it's lowest position.

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
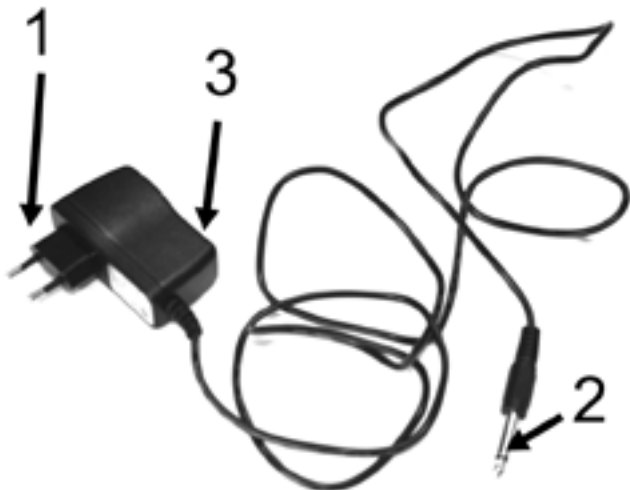
## CHARGING OF BATTERIES

To keep the full power of the battery as long as possible it is important to charge it regularly. Charging it once a week is often sufficient, as a suggestion at night when the chair is not in use. Never stop the charging until its completed. Charging can take place indoors in the living area without problems. When stored, the battery should be charged once a month to prevent it to go under the critical level when the charger cannot start. Do not leave the charging on for long continuous periods as this is not good for the battery. Don't leave the charger plug (2) in the socket (4) when charging is completed. It blocks the electric seat functions.

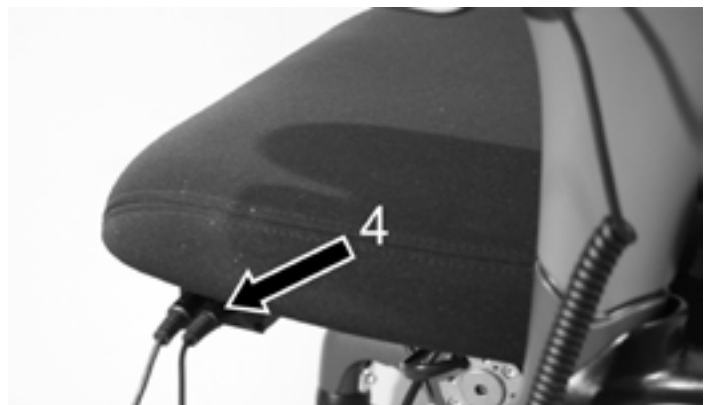1. Connect the charger plug (2) to the socket (4) on the chair. LED (3) lights green.
2. Connect the charger (1) to the wall socket. The LED lights red.
3. When charging is completed the LED (3) lights green.
4. Pull the charger(1) from the wall socket first and then the charger plug (2) from socket (4).

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## SERVICE

If the chair is used frequently, i.e. every day, it ought to have service regularly. This to control that the function and security of the chair is kept through all its lifetime.

If service is needed contact primarily your local dealer.

english

## WARRANTY

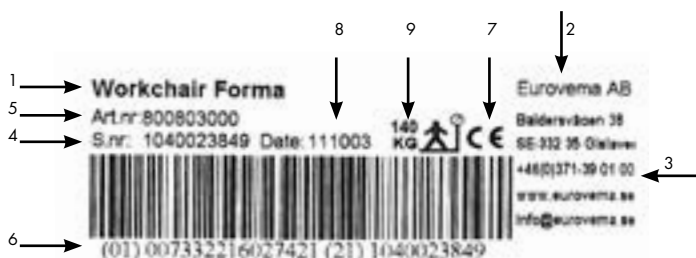
We give a 2 year warranty on our products for fabrication fault / damage. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

english

## CE - label

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. The label is found in front of the backrest holder. Tested and approved according to directive SSEN1335–1,2, 3. Firetested according to ISO7176-16

- 1) product
- 2) manufacturer
- 3) contact information
- 4) serial number
- 5) article number
- 6) EAN code
- 7) CE label
- 8) date of manufacturing
- 9) max user weight

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\*the seat lift position in the chassis is adjustable by loosening a screw joint. Approx: 3 cm down and 10 cm up. Affects min and max seat height.

**Eurovema**  
Creates mobility and ergonomics