<b>CZ</b>	<b>REFERENČNÍ PŘÍRUČKA</b>	<b>3</b>
<b>DE</b>	<b>GEBRAUCHSANWEISUNG</b>	<b>9</b>
<b>DK</b>	<b>BRUGSANVISNING</b>	<b>15</b>
<b>ES</b>	<b>MANUAL DE INSTRUCCIONES</b>	<b>21</b>
<b>FR</b>	<b>MODE D'EMPLOI</b>	<b>27</b>
<b>FI</b>	<b>KÄYTTÖOHJE</b>	<b>33</b>
<b>IT</b>	<b>MANUALE D'USO</b>	<b>39</b>
<b>NO</b>	<b>INSTRUKSJONSMANUAL</b>	<b>45</b>
<b>NL</b>	<b>GEBRUIKSAANWIJZING</b>	<b>51</b>
<b>PL</b>	<b>INSTRUKCJA OBSŁUGI</b>	<b>57</b>
<b>SE</b>	<b>INSTRUKTIONSHANDBOK</b>	<b>63</b>
<b>UK</b>	<b>INSTRUCTION MANUAL</b>	<b>69</b>

*Impulse S6-S8*

**© 2010 EXENDIS B.V.**

All rights Reserved. No part of this document may be photocopied, reproduced, stored in a retrieval system, or transmitted, in any form or by means whether, electronic, mechanical, or otherwise without the prior written permission of EXENDIS B.V.

No warranty of accuracy is given concerning the contents of the information contained in this publication. To the extent permitted by law no liability (including liability to any person by reason of negligence) will be accepted by EXENDIS B.V., its subsidiaries or employees for any direct or indirect loss or damage caused by omissions from inaccuracies in this document.

EXENDIS B.V. reserves the right to change details in this publication without notice.

<b>Company name</b>	: EXENDIS B.V.			
<b>Street address</b>	: Keesomstraat 4,	6716 AB,	Ede,	The Netherlands
<b>Postal address</b>	: P.O. Box 56,	6710 BB,	Ede,	The Netherlands
<b>Contact telephone</b>	: +31 (0)318 67 61 11			
<b>Fax number</b>	: +31 (0)318 67 61 03			
<b>Web address</b>	: <a href="http://www.exendis.com">http://www.exendis.com</a>			
<b>E-mail address</b>	: <a href="mailto:info@exendis.com">info@exendis.com</a>			

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## INSTRUCTION MANUAL BATTERY CHARGER Impulse S 6/8

You now have the instruction manual in front of you that comes with the **IMPULSE S** charger of EXENDIS B.V. with a rated current of 6 or 8 A. The **IMPULSE S** can charge gel batteries with 24 V. The **IMPULSE S** is a full automatic, computer controlled and safeguarded charger. The **IMPULSE S** has been designed with double insulation. We advise you to carefully read this instruction manual as well as to observe the Safety Instructions before taking the **IMPULSE S** into operation.

### 1. SYMBOLS

The followings show the symbols used.

	A triangle containing an exclamation mark indicates important information in these operating instructions which is to be strictly followed.
	Read instruction manual.
	Double insulation
	Apparatuses containing hazardous substances are marked by this symbol. This symbol also indicate that it is prohibited to dispose of these apparatuses in the household waste. You can return these apparatuses free of charge to the collection points in your community or to the supplier of the apparatus. <b>You thus fulfil the legal requirements and make your contribution to the protection of the environment!</b>
	This equipment is CE-tested and thus meets the EMC directive 2004/108/EG and the low-voltage directive 2006/95/EG.
	The GS-symbol (Geprüfte Sicherheit) is a German certification mark, indicating, that the apparatus satisfies the "Gerätesicherheitsgesetz" (law for safety of apparatus).

### 2. DESIGN / SPECIFICATIONS

Primary voltage	: 230 Vac - 50/60 Hz - 1-phase
Secondary nominal voltage	: 24 Vdc
Secondary maximum voltage	: 35 Vdc
Secondary current	: can be set by the supplier/dealer max. 6 A for Impulse S 6 max. 8 A for Impulse S 8
Battery type	: Gel
Battery capacity	:

	Min.	Max. (in order to charge 80%)
--	------	----------------------------------

		capacity within 8 hours)
Impulse S 6	40 Ah	60 Ah
Impulse S 8	60 Ah	85 Ah

Protective devices	: - protected against reversed polarity - protected against overvoltage - protected against high temperatures
Input power	: 195 W with Impulse S 6 270 W with Impulse S 8
Efficiency	: min. 80% (at full load)
Ambient temperature	: 0 °C to +40 °C
Dimensions cabinet	: Height 70 x Width 150 x Depth 200 mm
Safety class	: II
Overall weight	: 1.3 kg
Transport and storage	
Ambient temperature storage	: -15°C ... +50°C.
Relative air humidity max.	: 95 % (non condensing)

### 3. SAFETY INSTRUCTIONS

Carefully read the manual and the Safety Instructions. <b>Explosion hazard!</b> Avoid sparking or fire in the immediate environment of the battery
During charging the battery should be placed in a <b>well vented</b> place.
The <b>IMPULSE S</b> contains parts such as switches and relays that may produce sparks. If charging takes place in a garage or a similar place then the charger should be placed in such a position that sparking causes no risk.
<b>Never</b> use the charger in a demounted (open) condition. This may be life threatening if any parts under voltage are touched!
Note: It is <b>never</b> allowed to attempt to charge batteries that cannot be charged.
<b>Never</b> touch the battery clamps if the charger is in operation.
<b>Never</b> move the charger if it is in operation.
Do <b>not</b> use the charger in combination with a faulty cable or plug or if the charger itself is damaged.. If the mains lead needs replacing this must be done by the service department of your supplier/dealer
If the charger does not operate properly then you should first note which failure indication you observe, next you should remove the plug from the socket and subsequently contact the service department of your supplier/dealer.
<b>Never</b> attempt to demount or repair the charger yourself. In order to make sure that the repair is reliable and safe we advise you strongly to have any repairs executed by the service department of your supplier/dealer.
The charger may not be used by children or by people, who don't have experience or knowledge to use it. The user must have enough competency to install it. Otherwise he must ask for help by qualified personnel. This charger is not a toy for children.

If this safety instruction, as well as the instruction manual, is ignored then EXENDIS B.V. will not be liable for any failure, in addition a possible guarantee cannot be claimed.

## 4. BATTERY INSTRUCTIONS

Just **after some charge cycles** and usage, new batteries will get **full capacity**. Keep this in mind at the first trip.

After **every** trip, the batteries can be charged.

Charging **overnight** is possible.

Just when the green LED lights, the batteries may be disconnected from the charger. Only **if very necessary**, the charge cycle may be interrupted (at first disconnect the mains cable, at second disconnect the charge cable). In this case keep in mind, that the batteries are not full, and you can not ride a far distance.

If the green LED lights and the batteries are not disconnected, the battery is **kept in optimal full state**.

If batteries are not frequently used, it is recommended to **charge** the batteries **at least once a month** (due to the self-discharge of the batteries). It is also possible to let the batteries continuously connected to the charger.

Although the charger can charge deep discharged batteries, it is recommended **not to discharge batteries very deep**, because this shortens battery life. However, if this happens, it is recommended to charge the batteries as soon as possible in order to reduce battery damage.

The charger has been adjusted for only one battery type and capacity. If an other battery is connected to the charger, this battery can be damaged and/or dangerous situations may occur. Therefore if batteries are changed, you must contact your supplier/dealer, to adjust the charger to the correct battery type and capacity.

In case of any questions, you can contact your supplier/dealer

## 5. SETUP OF THE CHARGER

When placing the charger it should be observed that it has sufficient ventilation all around. A space of at least 10 cm should be allowed. When used under normal circumstances the built-in ventilator will be activated during charge (only Impulse S 8). If there is not sufficient ventilation or the temperature is high, the charger will lower the output current (resulting in longer charge time) or if the temperature rises over a maximum value, the charger will stop charging.

The cabinet is made of plastic and suitable as a table model.

During normal operation the housing is getting warm. This is normal.

## 6. CONNECTING CONDITIONS

### 6.1 Connection to 1-phase network

The charger is suitable for a mains voltage of 230V - 50/60Hz, 1-phase and has been fitted with a 1.9 meter mains lead and plug.

NOTE: Check whether the mains voltage is in accordance with the voltage the charger was designed for.

The mains plug should be plugged into a wall socket.

### 6.2 Connection charging cables

On the output (secondary) side the chargers have been fitted with a 2 meter 3-wire core charging cable. As the charger takes cable loss it is not allowed to replace this cable by a cable with a different length and/or diameter without adaptation by the supplier/dealer. Shortening the cable is therefor **not** allowed!

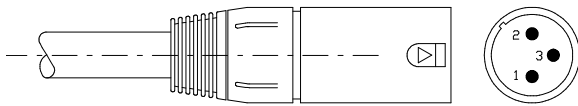
Figure 1 XLR-charging connector

The charging cable can be supplied together with an XLR-charging connector, whereby pin 1 of the XLR-charging connector is the positive pole (+) and pin 2 is the negative pole (-) and pin 3 is the drive inhibit signal (unless this is altered by your supplier/dealer).

In case the charging cable is supplied without a charging plug, this ought to be supplied together with the battery. When mounting this plug you should observe the polarity.

The positive of the battery is connected to the positive of the charger and the negative of the battery is connected to the negative of the charger. If the battery is connected to the charger with reversed polarity, the charger will not start charging and does not show any failure indication.

## 7. TAKING THE CHARGER INTO OPERATION

- The charger can now be connected.
- First of all you should plug the mains plug into the wall socket.
- The charger enters the '**STAND-BY**'-condition after flashing a combination of LED's.. Both LED's (yellow and green) are lighting.
- Subsequently you can connect the battery and charging will start automatically. Only the yellow LED is now lighting. At deep discharged batteries, first the battery is charged with a small current (precharge). After that the battery is charged with the nominal current (main charge). At last the battery is charged with a declining current (post charge). The magnitude of these currents depends on the adjusted battery type.
- When the yellow LED is switched off and the green LED on, the battery is full and can now be disconnected. If the battery is not disconnected then the battery is kept in optimal full state by a very low current.
- When the battery is disconnected, it is allowed to let the mains plug in the wall socket. The yellow and green LED's will light.

## 8. INDICATIONS

	Yellow LED 	Green LED 
Charger off (mains not connected)	○	○
Charger just switched on and showing the charge characteristics	⊙	○
	○	⊙
Stand-by	⊙	⊙
Charging	⊙	○
Full	○	⊙
Failure	⊙	⊙

○ = Off

⊙ = On

⊙ = Blinking

## 9. FAILURE REPORT INDICATIONS

If the charger does not function (anymore), it can be restarted by pulling the mains plug from the wall socket and putting it back in again. First of all you should however follow the indications in the next paragraphs based on the failure report indications.

### 9.1 Not a single LED is lighting

- There is no mains voltage.

First check whether the plug has been fitted in the wall socket. If this has been fitted in a wall socket it should be checked whether there is any voltage on the wall socket. If there is no mains voltage then you should check the safety fuses in the distribution cabinet or call a technician.

### 9.2 Both LED's light

If no battery is connected, this is a stand-by condition and no failure condition.

If a battery is connected however, one of the following failure conditions may have occurred:

- There is no battery voltage present.

Check whether the battery has correctly been connected to the charger.

If it is not connected then the connection must be restored and the charging process should be restarted.

- The polarity of the battery connected to the charger is not correct.

Check whether the polarity of the charger corresponds with the polarity of the battery.

If it is connected correctly then you should consult your supplier/dealer.

### 9.3 Only the green LED is blinking

If the yellow LED lights and the green LED is (slowly) blinking, (while the relay is audible switching), this means, that the charger starts charging. After that the charger sees that the voltage rises very fast en therefor stops charging and again starts charging. This usually means that the battery has been worn out and has to be replaced.

### 9.4 The yellow and green LED's are blinking

In case of a failure, both LED's blink a number of times followed by one second rest. This number indicates which failure has occurred.

Number of times the LED's blink followed by a short rest	Failure description	Remedy
4	Temperature too low or hardware problem	1)
5	Temperature too high	2)
1, 2, 3, 6, 7, 8	Other problem	3)

Remedy codes:

- 7) Move charger to a room with temperature above 0 °C and restart the charging process. If temperature is above 0 °C and the failure report repeats itself, then you should consult your supplier/dealer.
- 8) Allow the charger to cool down for some time and then the charging process should be restarted. If the failure report repeats itself then you should consult your supplier/dealer.
- 9) Consult your supplier/dealer.

## 10. CLEANING AND MAINTENANCE

The charger and the cables can be cleaned with a damp cloth. Do not use aggressive cleaning agents.

The charger is maintenance-free.

## 11. REPORTING SERVICE

When reporting the service department of your supplier/dealer you should always report the following data:

- i) Type of charger (IMPULSE S 6 / 8)
- j) Which LED's are lighting or which are not lighting?
- k) When did you observe the failure report?
- l) When was the charger taken into operation?

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**5.38.124.0615, 5.38.124.0816**

**5.38.124.0616**

**is conforming the following directives:**

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**The development and certificates have conformed the following standards /**

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**The manufacturing year and serial number have been printed on the device /**

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