



RESERVDELSLISTA Flexmobil 600 Comfort

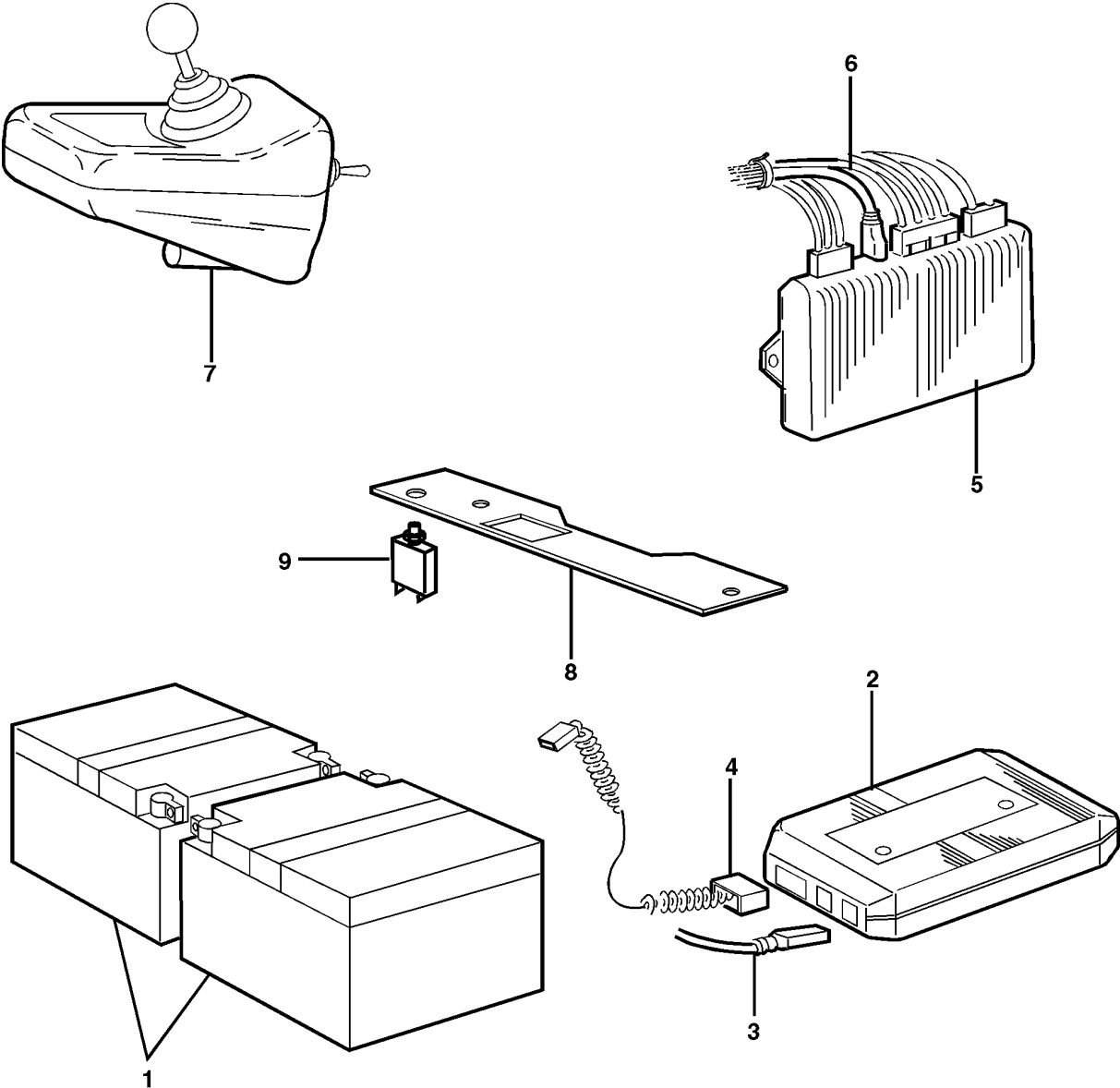

EUROFLEX
197
Printed in Sweden


RvS Euroflex AB
Box 1179
141 24 Huddinge
Tel: 08 449 58 00 Fax: 08 740 40 18

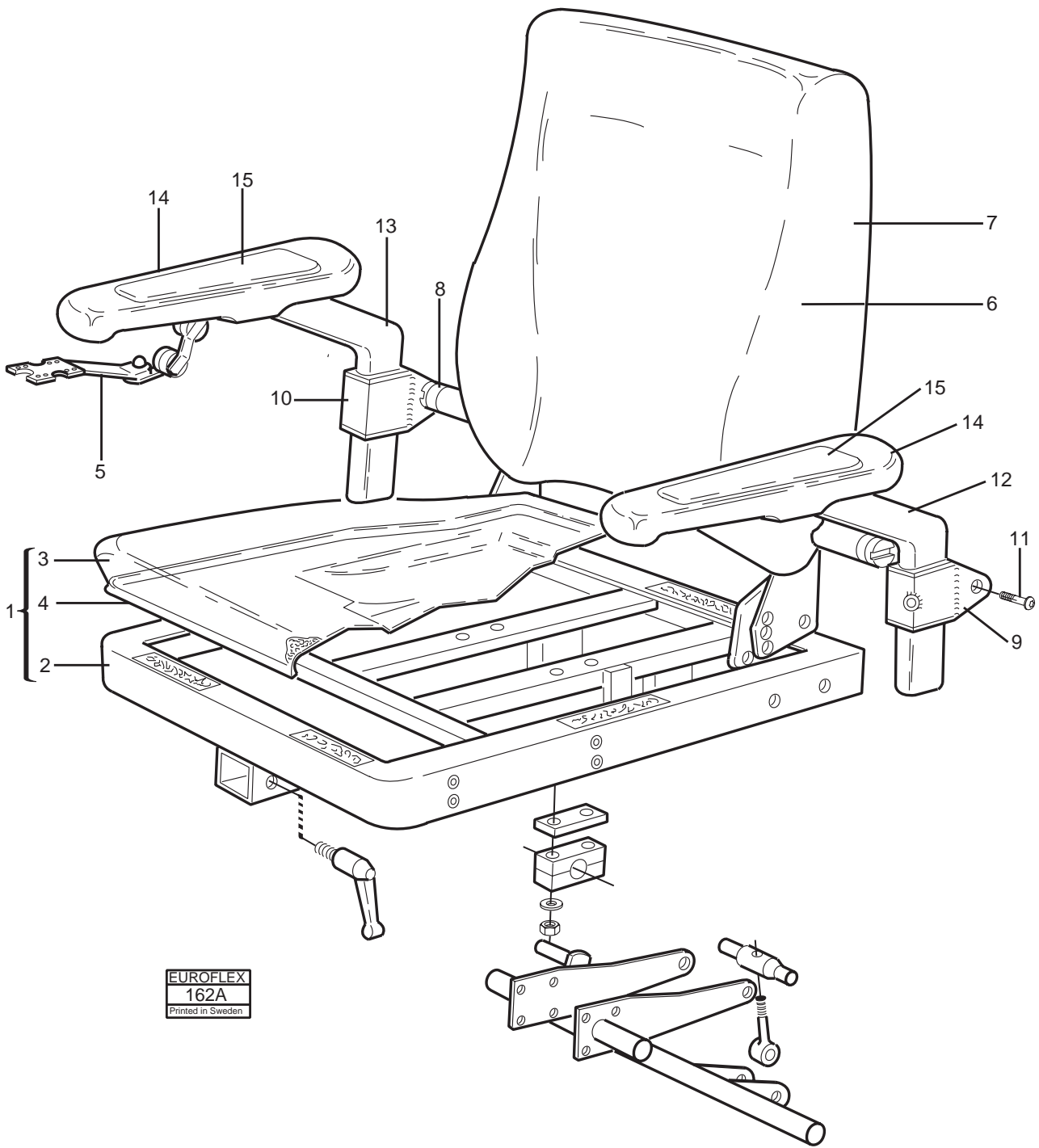

EUROFLEX
162A
Printed in Sweden


Utgåva Issue		Sitsenhet	Seat sushion		
1/98		Comfort	Comfort		
			Bild Nr Picture No	Grupp Group	Sida Side
			162A	2	6
Pos Fig	Art. Nr Part. No	Benämning	Description	Anmärkning Notes	
	299101-3000	Comfortsits, kmpl.	Comfort seat, cmpl.		
1	699101-3000	. Ram, kmpl.	. Frame, cmpl.		
2	357001-6000	.. Ram	.. Frame		
3	699101-3002	.. Sitsskum	.. Sekt foam		
4	699101-3001	.. Plastplatta	.. Plastic plate		
5	699201-7900	. Fäste	. Joystick fixation		
6	699101-6001	. Ryggskum	. Backrest foam		
7	699101-6000	. Rygggram	. Back rest frame		
8	624001-9000	. Kugglås	. Angle adjustment		
9	699001-4001V	. Fäste	. Armrest fixation	V / LH, Angle adjustable H / RH, Angle adjustable	
10	699001-4001H	. Fäste	. Armrest fixation		
11	7K6S-8x35	. Skruv	. Screw	K6S M8x35	
12	699001-4002V	. Fäste	. Armrest fixation	V / LH, Hight adjustable H / RH, Hight adjustable	
13	699001-4002	. Fäste	. Armrest fixation		
14	624001-4000	. Armstöd	. Arm rest pad		
15	624001-4001	. Illägg	. Arm rest insert	Tyg / Fabric	

RESERVDELSLISTA

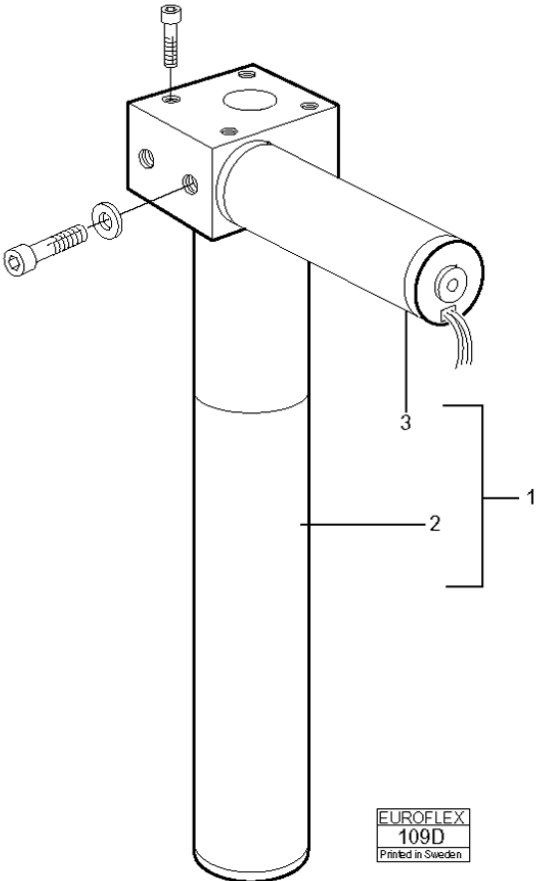

RvS Euroflex AB
Box 1179
141 24 Huddinge
Tel: 08 449 58 00 Fax: 08 740 40 18

RESERVDLSLISTA

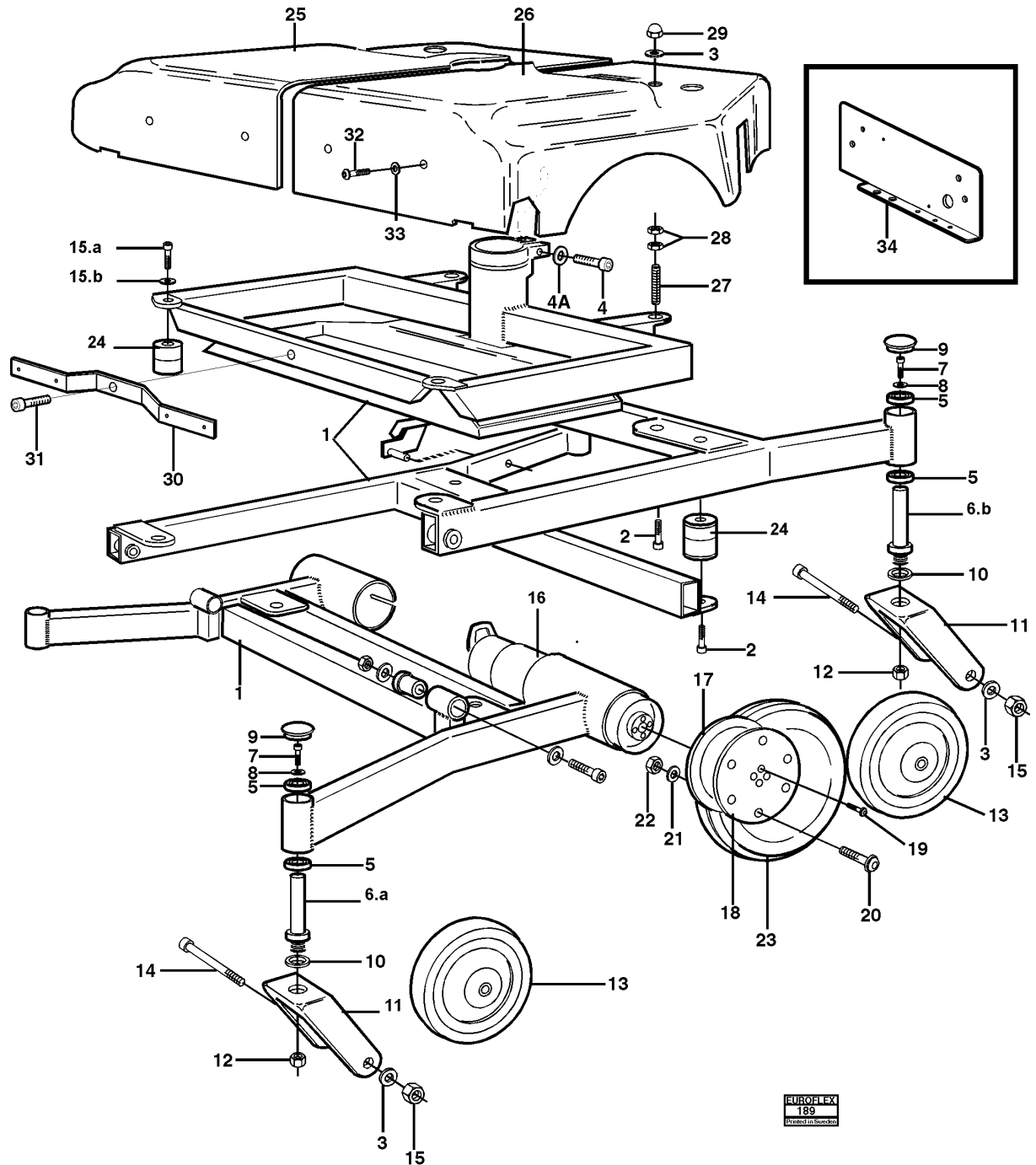

EUROFLEX
189
Printed in Sweden

RvS Euroflex AB
 Box 1179
 141 24 Huddinge
 Tel: 08 449 58 00 Fax: 08 740 40 18

RESERVDDELSLISTA

UTGÅVA ISSUE 1/01		FLEXMOBIL600 CHASSI	FLEXMOBIL600 CHASSIS	BILD NR. 189 PICTURE NO.
Pos. Fig.	Art.nr Part.nr	Benämning	Description	Anmärkning Notes
1.	669101-1000	Chassie 600 serie vuxen	Chassis 600 adult	
2.	7mc6s-6x16	Skruv M6x16	Screw M6x16	MC6S-6x16
3.	7brb-8,4x16	Planb. 8,4x16x1,6	Washer 8,4x16	
4.	7mc6s-8x35	Skruv M8x35	screw M8x35	MC6S-8x35
4.a	7fbb-8,1	Fjäderbricka M8	Spring washer	FBB 8,1
5.	7k6001-2z	Kullager 6001-2z	Ball bearing 6001 2z	
6.a	669001-1102	Fästtapp alu-bygel,låg	Anchor pin	
6.b	669001-1101	Fästtapp alu-bygel,förhöjd	Anchor pin extra tall	
7.	7mc6s-6x16	Skruv M6x16	Screw M6x16	MC6S-6x16
8.	7brb-M6x18	Planbricka M6x18x1,6	Washer 6x18x1,6	
9.	7sfl-32	Ändavslutare	Plastic plug	
10.	Planb.brb	Planbricka 12x22x2	Washer 12x22x2	
11.	669201-7000	Fram/bak-länkhjulsgaffel	Castor front/rear	
12.	7din-12x1,25	Mutter Din M12x1.25	Nut M12x1,25	
13.	7hjul-125	Hjul 125 grå bana,nav38	Wheel 125mm gray	
14.	7mc6s-8x60	Skruv M8x60 fzb 8.8	Screw M8x60fzb 8.8	MC6S-8x60
15.	7kl-6-8	Låsmutter kl6 M8	Nut M8	
16.a	269001-1110h	drivmotor+fälg+däck hö.	Drive unit+ wheel/tyre	Right
16.b	269001-1110v	Drivmotor+fälg+däck vä.	Drive unit+wheel/tyre	Left
17.	669201-7400	Fälgsida inre	Rim inner side	
18.	669201-7300	Fälgsida yttre	Rim outside	
19.	7f6s-5x12	Försänkt ins M5x12obeh.	Screw M5x12	F6S-5x12
20.	7mf6s-5x16	Skruv M5x16 obehandlad	Screw M5x16	MF6S-5x16
21.	7brb-5,3x10	Planbricka M5x10x1	Washer 5x10x1	
22.	7kl-6-5	Låsmutter kl6 M5 fzb	Nut M5	KL-6-5
23.	769201-7000	Däck 2.00-4 (225x50)	Tyre 2.00-4 (225x50)	
24.	7vibra-8x40	Vibrationsdämpare	Vibration absorber	
25.	669101-9000h	Kåpa delad höger	Cover right side	
26.	669101-9000v	Kåpa delad vänster	Cover left side	
27.	7pinnb-8x51	Pinnbult M8x51	Stud bolt M8x51	
28.	7ml6m-8 låg	Mutter M8-låg fzb	Nut M8	ML6M-8 low
29.	7mhm-8	Kupolmutter M8 fzb	Nut M8	MH6M-8
30.	669001-1005	Kåpfäste	Fixation plate /cover	
31.	7mc6s-6x16	Skruv insex M6x16	Screw M6x16	MC6S-6x16
32.	7f6s-6x16	Förs.insex M6x16 fzb	Screw M6x16	F6S-6x16
33.	7brb-12x22	Planbricka 12x22x2 fzb	Washer 12x22x2fzb	BRB-12x22
34.	669001-1001	Fästplåt PG-powermodul	Mounting plate / PM	

RvS Euroflex AB
Box 1179
141 24 Huddinge
Tel: 08 449 58 00 Fax: 08 740 40 18