

Eurovema

Creates mobility and ergonomics.



Assembly Instruction

- Folding tray, adult/abc

Eurovema

Baldersvägen 38, SE-332 35 Gislaved, Sweden

Tel./Phone +46 371 390 100

info@eurovema.se | www.eurovema.se

ASSEMBLY OF TRAY, FOLDING, ADULT/ABC


- 1) Start by lifting the armrest out of the width adjustment.




- 2) Place the tray bracket on the height adjustment.



- 3) When the tray is at the correct height, tighten the stop screw using an Allen key number 4.

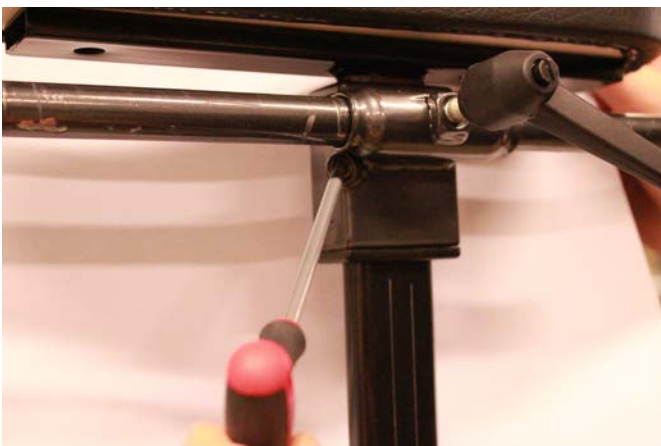

- 4) Thread the steel tube into the tray bracket, put on the end cap.

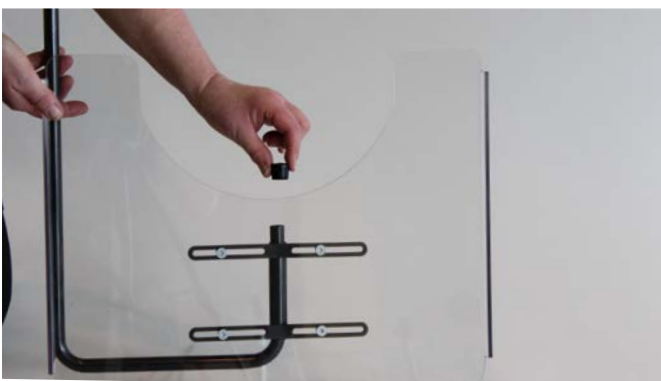




- 5) Insert the tray into the bracket on the armrest.



- 6) To lock the tray in the right position, tighten the lever.



- 7) Put on the end cap.

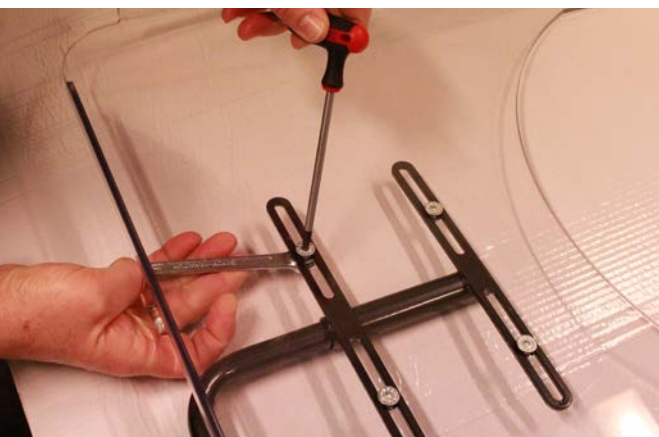

- 8) To adjust the tray laterally, use Allen key number 4 for the screw and a number 10 spanner for the nut.



Eurovema

Creates mobility and ergonomics.

Baldersvägen 38, SE-332 35 Gislaved, Sweden
Tel./Phone +46 371 390 100 | info@eurovema.se | www.eurovema.se