

Reconditioning directions, instruction and service for Volaris rollators

Wheels

Check that/if	To do	Comments	OK
The wheels are undamaged and the ball bearing does not sound bad	The wheel must run easily otherwise replace wheel (bearing in poor condition)	On wheel change, also replace locking nuts	
The treads are not damaged	Replace wheel	New locking nuts	
Check that the screws are not loose	Tighten the screws (use glue, Loctite 270)		
If locking nuts are missing	New locking nuts (use glue, Loctite 270)		
The front fork is properly secured in the upright	Tighten the front fork properly with new locking nuts (or use glue, Loctite 270)		
The front fork is undamaged	Replace the front fork		

Brakes

Check that/if	To do	Comments	OK
That the brake shoulder is not worn	Replace brake shoulder		
Distance between brake shoulder and wheel is 1 mm	Adjust		
Check that the brakes work properly Brake handle is undamaged	Adjust, lubricate cable Replace brake handle		
Check that the brake handle is properly secured in the aluminium profile	Tighten the screws, replace brake handle if needed		
The handle height adjustment is easy	Change profile		
The profile is not damaged	Change profile		

Folding mechanism

Check that/if	To do	Comments	OK
Folding is easy	Lubricate with a little grease on cross		
Folding catch closes easily and locks	Replace folding catch		
Damage on the cross	Replace the cross		

Handle

Check that/if	To do	Comments	OK
The handle is undamaged	Replace the handle grip		
The handle armrest pads are undamaged	Replacing the armrest pad		
Check that the screws are not loose	Tighten the screws (use glue, Loctite 270)		

Adjustments

Check that/if	To do	Comments	OK
Check that the height adjustment of the handle locks properly	Replace lock mechanism		
Locking knob for height adjustment	Tighten the locking knob for height adjustment as well as locking profile and flat washer.		
That the seat can be adjusted and locks	Replace seat		

Frame

Check that/if	To do	Comments	OK
No mechanical damage on the upright	Scrap the rollator		
The paintwork is not damaged	Repair damage with touch-up paint		

Seat

Check that/if	To do	Comments	OK
Seat is undamaged	Replace seat		

Back support band

Check that/if	To do	Comments	OK
Back support/attachment is not damaged and the screws are tightened properly	Replace back support and tighten screws		

Reconditioning

Check that/if	To do	Comments	OK
---------------	-------	----------	----

Leave the brakes off when washing the rollator in a washing unit. The rollator must cool down to room temperature before it is used and adjusted.

Wash cabinet in 60 degrees for 45 min

For manual wash: Wash your rollator with a wet cloth and a mild cleaning liquid. Ensure that the gaps between the wheels and the brake shoes are free of dirt; dirt here can reduce the braking effect! You can rinse off by using a high-pressure cleaner or garden hose.

Dry it off afterwards

Rollator model: _____

Serial number: _____

Service performed by

Name: _____

Date: _____

Signature: _____