





ALVEMA  
173  
Printed in Sweden

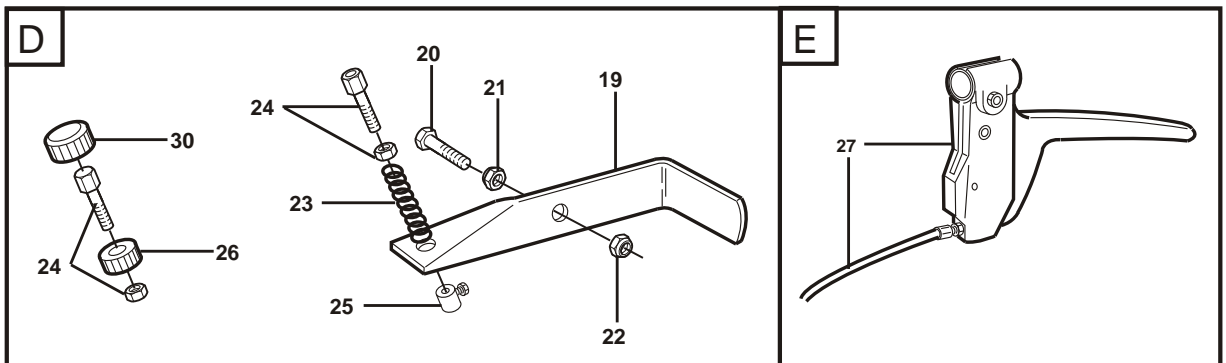




# RvS ALVEMA

## Atila Reservdelar / Spare parts

Utg./ Issue	2005-01-12 Rev A	Typ av produkt	Type of product	
Bild/pic	nr. 173	Atila 1020 7000,7001,7002	1020 7000,7001,7002	
Pos	Art.nr:	Benämning:	Description:	number
Fig	Part no:			Antal
1	370004-2210	främre kryss	front frame	1
2	370005-1310	bakre kryss för hjul	rear frame for wheels	1
3	370004-1310	bakre kryss doppsko	rear frame for rubber top	1
4	7707-1000	doppsko 22 mm	rubber top 22 mm	2
5	7MC6S-8*40	skruv	screw	2
6	7locking M8L	låsmutter	locking bolt	2
7	7IKP-22*19	ändavslutn.	end cap	2
8	7CT476-30*16	vingvred	knob	2
9	370004-1000	tvärrör	metal bar	1
10	3700-3100	handtag	handle	2
12	7000-2000	plastkrok svart glasfib.	plastic hook	1
13	340001-9200	distanshylsa	washer	1
14	7KHN-6*24	hålnit	rivet	1
15	76666-6	starlock m kapsel 6 mm	starlock w. capsule 6 mm	1
16	72224-0007	bussning plast	plastic knob	2
17	7pop-4,8*16	popnit	pop rivet	2
18	7-5*13*22	stapelskydd m radie	pileprotection	2
19	3700-1210 H	broms höger svart	brake right black	1
19	3700-1220 V	broms vänster svart	brake left black	1
20	7 MCS-6*40	spårskruv	screw	2
21	7M6M-M6	mutter	nut	2
22	7locking M6L	låsmutter	lockingnut	2
23	7FJT-9*50*1	tryckfjäder	spring	2
24	776-21-541	just.skruv+mutter	adj.screw + nut	4
25	758-25-150	skruvnippel m skruv	screw	2
26	7RAMU-375M5	ratt	knob	2
27	670001-3100	bromshandtag kompl.	brake handle w. holder	2
28	7000-1000	handtag cellgummi svart	soft grip black	2
29	7MC6S-M8*55	skruv	MC6S	2
30	7RAMU-376M6	ratt	knob	2
33	7MC6S-M8*55	skruv	screw	2
34	670001-5000	låsring	locking ring	1
35	7IKP-20*14	ändavslutning	end cap	2

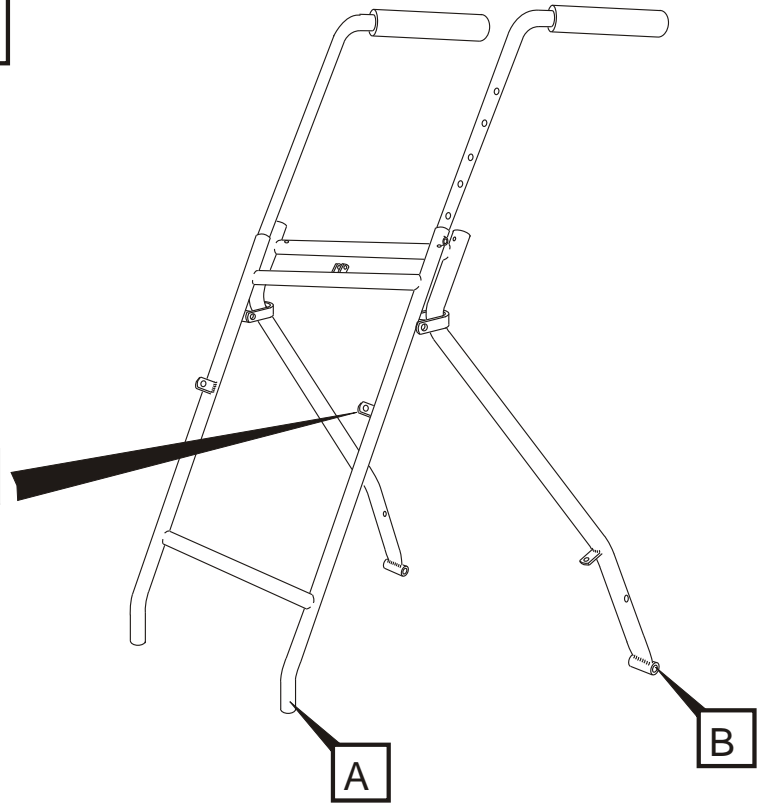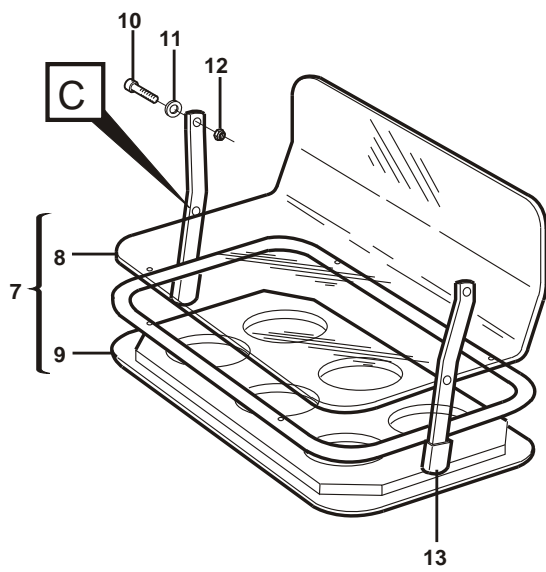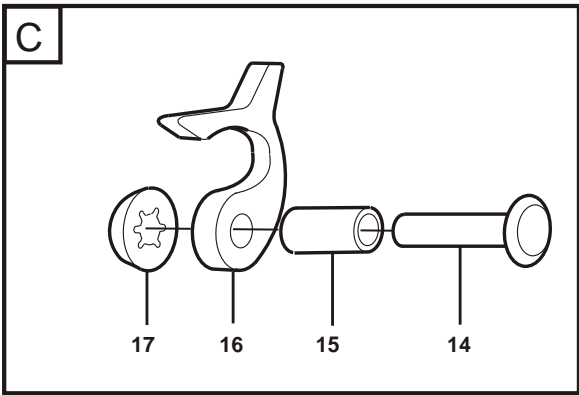

ALVEMA  
174  
Printed in Sweden

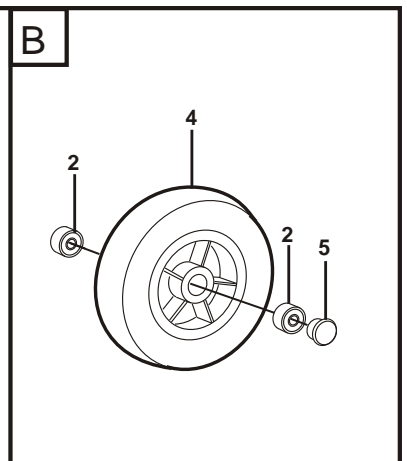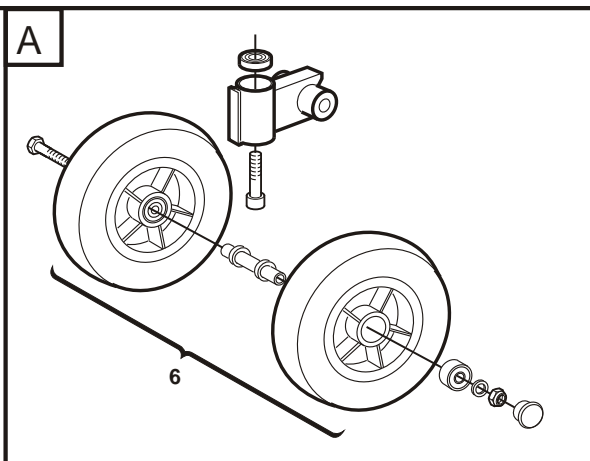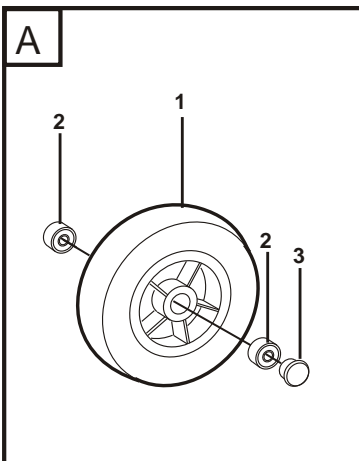

# RvS ALVEMA

## Atila Reservdelar / Spare parts

Utg./Issue Bild/pic	2005-01-12 Rev A nr 174	Typ av produkt Atila 1020 7000,7001,7002	Type of product 1020 7000,7001,7002	
Pos Fig	Art.nr: Part no:	Benämning:	Description:	Number Antal
1	611001-7100	hjul 150*30	wheel 150*30	2
2	7BRB-8,4*16	bricka	washer	2
3	76671-8	starlock	starlock	2
4	67001-7100	bakhjul	rear wheel	2
5	76675-12	starlock	starlock	2
6	2700-7100	dubbelhjul	double wheel	2
7	60101570	bord	table	1
8	670801-1000	plastplatta	plastic plate	1
9	60 102 670	sitsplatta	seat	1
10	7M6S-6*16	skruv	screw	2
11	7BRB-6,4*12	bricka	washer	2
12	7locking-M6L	låsmutter	locking nut	2
13	7-4*18*30	plaststopp	plastic stop	2
14	7KN-6*20	nit	rivet	1
15	340001-9200	distans	distance	1
16	7000-2000	krok	hook	1
17	76666-6	starlock	starlock	1